

SELENIA[®]
Dimensions[®]



User Guide

MAN-06485-002 Revision 001

HOLOGIC[®]

Technical Support

USA:	+1.877.371.4372
Europe:	+32 2 711 4690
Asia:	+852 37487700
Australia:	+1 800 264 073
All Other:	+1 781 999 7750
Email:	BreastHealth.Support@hologic.com

© 2019 Hologic, Inc. Printed in the USA. This manual was originally written in English.

Hologic, 3D, 3D Mammography, Affirm, C-View, Dimensions, FAST Paddle, Genius, H-View, ImageChecker, Quantra, Selenia, SmartCurve, and associated logos are trademarks and/or registered trademarks of Hologic, Inc., and/or its subsidiaries in the United States and/or other countries. All other trademarks, registered trademarks, and product names are the property of their respective owners.

This product may be protected by one or more U.S. or foreign patents as identified at www.Hologic.com/patents.

Selenia Dimensions System User Guide

Table of Contents

Figure 43: An Example Custom Output Group.....	71
Figure 44: Select the Images for Export.....	73
Figure 45: Export Dialog Box.....	73
Figure 46: Print Screen.....	75
Figure 47: Image Display Screen.....	77
Figure 48: Exposure In Progress.....	79
Figure 49: Exposure Complete.....	79
Figure 50: Marked Images in a Procedure.....	80
Figure 51: Tools Tab (Tomosynthesis option shown).....	82
Figure 52: Image Review Tools.....	83
Figure 53: Tools on the Notices Tab.....	84
Figure 54: Exposure Index.....	85
Figure 55: Display Modes.....	85
Figure 56: Slice Indicator.....	86
Figure 57: I-View 2D Contrast Screen.....	87
Figure 58: I-View 2D Contrast Screen, Waiting Period.....	88
Figure 59: I-View 2D Contrast Screen, Optimal Imaging Period.....	89
Figure 60: I-View 2D Contrast Settings.....	90
Figure 61: Procedure Screen for Specimens.....	91
Figure 62: Specimen Imaging Screen.....	92
Figure 63: C-arm Accessories.....	93
Figure 64: Align the Retractable Face Shield on the C-arm.....	94
Figure 65: Face Shield Installation.....	95
Figure 66: Face Shield Operation.....	95
Figure 67: How to Install the Conventional Face Shield.....	96
Figure 68: How to Install a Compression Paddle.....	101
Figure 69: How to Remove a Compression Paddle.....	101
Figure 70: The FAST Compression Mode Slide.....	103
Figure 71: Installation of the Magnification Stand.....	104
Figure 72: Installation of the Localization Crosshair Device.....	105
Figure 73: How to Install and Remove the Magnification Crosshair Device.....	107
Figure 74: Example of a Screening Procedure Screen.....	110
Figure 75: Exposure In Progress.....	112
Figure 76: Exposure Complete.....	112
Figure 77: Calculating needle depth.....	114
Figure 78: Admin Screen.....	121
Figure 79: System Tab of the About (the Acquisition Workstation) Screen.....	123
Figure 80: Licensing Tab of the About Screen.....	124
Figure 81: Enable Auto-Hanging and Auto-Pairing.....	125
Figure 82: Enable Multi Line Procedure Tabs.....	126
Figure 83: My Settings Button in the Admin Screen.....	127
Figure 84: Console Tab of the Edit Operator Screen.....	127
Figure 85: Height Adjust Control Panel.....	128
Figure 86: Desired Console Height and Current Console Height Fields.....	128
Figure 87: Set Image Auto Disposition.....	129

6.7 How to Use I-View 2D Contrast



Note

Hologic configures some systems to meet specific requirements. Your system configuration may not have all the options and accessories included in this manual.

1. Select the **Contrast** tab to access the **I-View™ 2D Contrast (CEDM)** function.



Figure 57: I-View 2D Contrast Screen

2. Set the contrast settings. For more information, refer to [How to Configure the Contrast Settings](#) on page 90.



Note

Contrast information is included in the DICOM header of I-View images.