

# spiderX tech™

**A professionally engineered dynamic therapeutic support for muscles.**

- Advanced therapeutic tape technology
- Supports strained muscles in an active, dynamic way
- Enhanced muscle activation
- Improved performance
- Improved feedback and timing of muscles
- Prevention of further injury
- Protection during healing
- Improved circulation and lymphatic flow

## For Therapeutic Applications


SpiderTech Pre-Engineered Kinesiology Taping Applications are water-repellent elastic and adhesive sports tape appliances that can be worn on the skin for several days. Made out of cotton fabric and an acrylic adhesive activated by friction post-application, SpiderTech Applications minimize residue post-removal and contains no latex.

### SpiderTech Application:

- Clean and dry skin before application. Removal of hair in application area is recommended
- Extend or stretch the muscles/skin of the area being taped, then apply the tape un-stretched
- For specialized therapeutic methods, the tape may be stretched before application
- To remove, pull in the same direction as hair growth

### Attention:

- Do not apply onto injured or irritated skin
- Keep out of reach of Children
- Store at 41°F to 86°F, do not expose to direct sunlight
- For single use only
- Colors of tapes may differ from printed Picture on pack
- Specifications may change without notice
- Read the entire package insert before using

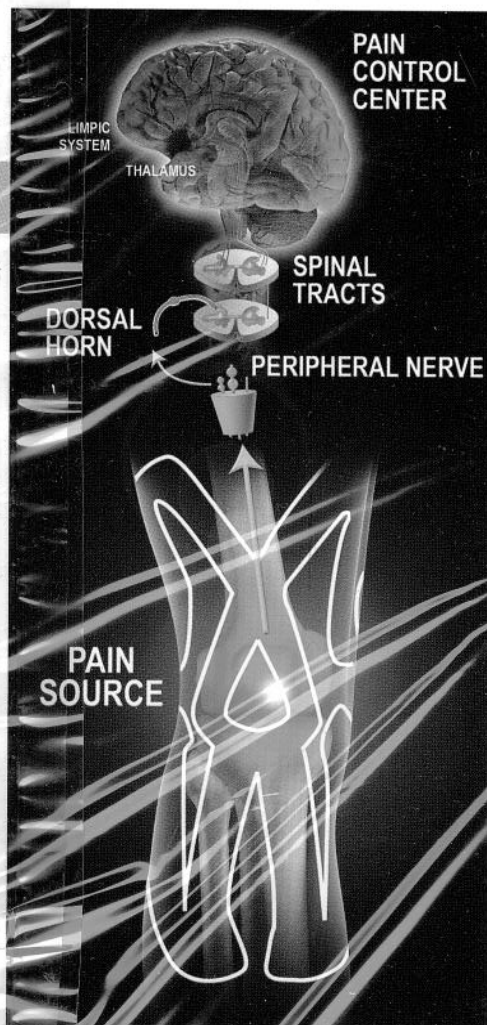

Patents and design patents pending



For complete instructions visit  
[www.nucapmedical.com](http://www.nucapmedical.com)



Produced by:  
NUCAP Medical Inc.  
3370 Pharmacy Avenue,  
Toronto, Ontario  
Canada M1W 3K4

Tel: (416) 490-4000  
Fax: (416) 490-4001

[info@nucapmedical.com](mailto:info@nucapmedical.com)  
[www.nucapmedical.com](http://www.nucapmedical.com)

SPIDERTECH and UPPER KNEE SPIDER are trade-marks of Nucap Medical Inc.

The original Kinesiology Tape from Japan

NITTO DENKO