Clean™ is specially formulated for use in washer disinfectors. Compatible with a variety of substrates, stainless, aluminum, and plastics. Extremely free rinsing. Strong chelating additives helps protect instruments by preventing scale buildup, allowing hard water minerals to be completely rinsed away.

For heavily soiled or stained stainless steel surgical instruments, use Clean™ undiluted to help in the removal of bioburden and stains.

For heavily soiled instruments a pretreatment with Treat™ Enzymatic Pretreatment should be used for the rapid breakdown of bioburden.

#### Directions For Use

Clean™ can be used in mechanical washing systems by injecting into the wash cycle. Typical usage 1/4 to 1 ounce per gallon of water (2 to 8 mL per liter).

Clean™ Mild Alkaline Instrument Detergent Concentrate is not a sterilant. Instruments must be sterilized following treatment with this product.

#### Caution. For External Use Only. Do Not Use Near Eyes. Keep Out Of Reach Of Children

In case of eye contact, flush with water for 15 minutes. Assure adequate flushing of eyes by separating the eyelids with finger. If irritation persists, get medical attention. In case of accidental ingestion, seek medical attention or contact a poison control center immediately. For skin contact, wash affected area immediately with water. Please refer to the MSDS sheet for all warnings and precautions for this product.

# CLEAN

## Mild Alkaline Instrument Detergent Concentrate

Quickly Removes Bioburden

Extends Instrument Life

Formulated for Mechanical Washers

**CAUTION:** Keep out of reach of children.  
See side panel for precautionary information

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Div. of PDQ Manufacturing, Inc.