

Operator's Manual

Life Scope G9 Bedside Monitor CSM-1901

CSM-1000 series



About This Manual

In order to use this product safely and fully understand all its functions, read this manual before using the product. Keep this manual near the instrument or in the reach of the operator and refer to it whenever the operation is unclear.

Accompanying Documentation

The bedside monitor comes with the following manuals. Refer to the manual depending on your needs.

Operator's Manual (this manual)

Describes the operation and settings of the bedside monitor. Read this manual before use.

Administrator's Guide

For administrators. Describes how to install the bedside monitor. Read the Operator's Manual together with this guide.

Service Manual

For qualified service personnel. Describes information on servicing the bedside monitor. Only qualified service personnel can service the bedside monitor.

Copyright Notice

The entire contents of this manual are copyrighted by Nihon Kohden. All rights are reserved. No part of this document may be reproduced, stored, or transmitted in any form or by any means (electronic, mechanical, photocopied, recorded, or otherwise) without the prior written permission of Nihon Kohden.

Trademark

The company name and model name are trademarks and registered trademarks of each company.



The mark printed on the SD card that is used in this instrument is a trademark. The company name and model name are trademarks and registered trademarks of each company.

This product stores personal patient information. Manage the information appropriately.

Patient names on the screen shots and recording examples in this manual are fictional and any resemblance to any person living or dead is purely coincidental.

The contents of this manual are subject to change without notice. If you have any comments or suggestions on this manual, please contact us at: <https://www.nihonkohden.com/>

Contents

1 Bedside Monitor Overview

General.....	1-2
Basic Operation.....	1-6
Multiple Displays.....	1-7
Menu Window.....	1-8
Home Screen.....	1-9
Symbols.....	1-10

2 Description of Parts

Core Unit.....	2-2
Display Unit.....	2-5
Input Units	2-6
Recorder Unit.....	2-17
RY-910PA Remote Controller.....	2-18
Determination of Applied Parts.....	2-19

3 Safety Information

Precautions.....	3-2
Warnings and Cautions	3-5
Installation and Connection	3-26
General Requirements for Connecting Medical Electrical Systems.....	3-31

4 Preparation

Loading Recording Paper.....	4-2
Inserting and Removing the Input Unit from the Data Acquisition Unit.....	4-3
Preparing the Remote Control.....	4-5
Backup Battery	4-10
Turning the Bedside Monitor and Display On.....	4-13
Turning the Bedside Monitor and Display Off.....	4-15
Clock Accuracy.....	4-16
Changing Settings	4-17

5 Admitting or Discharging a Patient

Admitting a Patient.....	5-2
Discharging a Patient.....	5-4

6 Alarm Function

General.....	6-2
Deactivating Alarms.....	6-7
Changing Alarm Settings.....	6-11
Setting All Alarms to Alarm Master Settings	6-16
Interbed Alarm	6-17
Changing Alarm Sound Volume.....	6-18
Alarm Recording.....	6-18

7 Monitoring

Monitoring Overview.....	7-0-2
Home Screen.....	7-0-4

7-1 Basic Parameters

ECG Monitoring.....	7-1-3
SpO ₂ Monitoring with Nihon Kohden Probes.....	7-1-16
SpO ₂ Monitoring with Nellcor Probes	7-1-26
SpO ₂ Monitoring with Masimo Probes	7-1-35
Respiration Monitoring.....	7-1-46
CO ₂ Monitoring	7-1-50
NIBP Measurement.....	7-1-62
IBP Monitoring.....	7-1-76
Temperature Monitoring.....	7-1-88

7-2 Other Parameters

BIS Monitoring.....	7-2-3
Gas Monitoring.....	7-2-10
Flow/Paw Monitoring	7-2-16
O ₂ Monitoring.....	7-2-20
EEG Monitoring	7-2-26
CO Measurement.....	7-2-35
TOF Monitoring.....	7-2-44
rSO ₂ /StO ₂ Monitoring.....	7-2-54

7-3 External Instrument Parameters

CCO Monitoring (CCO Monitor)	7-3-3
CCO Monitoring (PiCCO Monitor).....	7-3-8
PCCO Monitoring (APCO).....	7-3-12
PCCO Monitoring (Hemodynamic Unit).....	7-3-15
Anesthesia Monitoring.....	7-3-22
Ventilation Monitoring	7-3-29
tcPO ₂ /tcPCO ₂ Monitoring.....	7-3-41
Analog Input Monitoring.....	7-3-43

Incubator Monitoring..... 7-3-44

8 Review

General..... 8-3
Trend Graphs 8-6
Vital Sign List..... 8-12
Full Disclosure..... 8-18
ST Recall..... 8-23
Alarm History..... 8-27
Arrhythmia Recall..... 8-29
ECG Analysis Results..... 8-34
Hemodynamics Graph..... 8-42
PK Simulation Graphs 8-48
EEG Trend..... 8-58
OCRG..... 8-64

9 Analysis and Calculation

General..... 9-2
ECG Analysis..... 9-3
Drug Calculation..... 9-6
Lung Function Calculation..... 9-9

10 Recording

General..... 10-0-2
Changing the Recording Settings..... 10-0-7

10-1 Real Time Recording

Manual Recording 10-1-2
Alarm Recording..... 10-1-4
Periodic Recording 10-1-5
ECG Recording 10-1-6

10-2 Review Recording

Recording Trend Graphs 10-2-2
Recording the Vital Sign List..... 10-2-6
Recording the NIBP List..... 10-2-8
Recording the Hemodynamics List..... 10-2-10
Recording the Lung Function List..... 10-2-12
Recording the TOF List..... 10-2-14
Recording the Full Disclosure Waveforms 10-2-15
Recording the ST Recall Waveforms..... 10-2-17
Recording the ST Review Data 10-2-18
Recording the Alarm History..... 10-2-19
Recording the Arrhythmia Recall Waveforms... 10-2-21
Recording the Arrhythmia Event Graphs 10-2-23
Recording ECG Analysis Results 10-2-24

Recording Hemodynamics Graphs..... 10-2-32
Recording the EEG Trend 10-2-34
Recording the OCRG Review Data 10-2-36

10-3 Other Recording

Recording OCRG on the Home Screen 10-3-2
Recording the Flow/Paw Data and Loop
Waveforms 10-3-4
Recording the CO Measurement Results..... 10-3-4
Recording the TOF Measurement Results 10-3-5
Recording the Analog Waveforms 10-3-6
Recording the ECG Analysis 10-3-7
Recording the Drug Calculation..... 10-3-9
Recording the Lung Function Calculation..... 10-3-10

11 Interbed

General..... 11-2
Interbed Alarm Function 11-6
Changing the Interbed Alarm Settings..... 11-7
Advanced Interbed 11-8

12 Transport and HiQ-View Functions

Transport Function 12-2
Saved Data..... 12-5
Sending Data to the Central Monitor..... 12-7
Patient Authentication..... 12-8
Preparing for Input Unit Removal 12-10
Standby Status 12-11
WLAN Transport..... 12-12
HiQ-View Function..... 12-17

13 Troubleshooting

Screen Messages..... 13-3
Troubleshooting..... 13-41

14 Maintenance

Daily Check 14-2
Expiration, Replacement and Disposal..... 14-5
Cleaning, Disinfection and Storage 14-11
Periodic Inspection 14-26
Repair Parts Availability Policy 14-26

15 Factory Default Settings

Factory Default Settings 15-2

16 Standard Accessories, Options and Consumables

Standard Accessories and Options 16-2

Consumables..... 16-10

17 Specifications

Specifications 17-2

Socket Pin Assignment..... 17-78

Index

1

2

3

4

5

6

7

7-1

7-2

7-3

8

9

10

10-1

10-2

10-3

11

12

13

14

15








16

17

Unpacking

Check that all the items are included in the package. If there are any missing items, contact your Nihon Kohden representative.

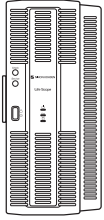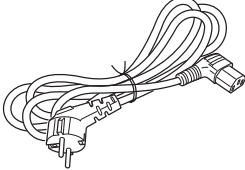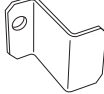The name and quantity are described under the illustration.

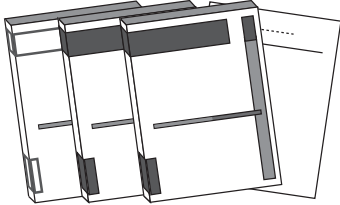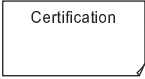


 <p>CU-191R or CU-192R core unit (1)</p>	 <p>Example is power cord UL Power cord (20m) (1)</p>	 <p>VL cable stopper (1)</p>
 <p>WLAN cable stopper (1)</p>	 <p>Screw F3 x 6 (3)/Screw PS 3 x 8 (2)</p>	 <p>Manuals</p>
 <p>Certification (1)</p>		

For the standard accessories such as power cord and for consumables such as recording paper, only use Nihon Kohden specified parts and accessories to satisfy the safety and performance.



For the model, code number and supply code of the standard accessories, refer to the "Options and Consumables" page of Section 16 "Standard Accessories, Options and Consumables" in this manual.

Conventions Used in this Manual and Instrument

Dangers, Warnings and Cautions

Level	Description
 DANGER	A danger alerts the user to a hazardous situation which causes death or serious injury.
 WARNING	A warning alerts the user to possible injury or death associated with the use or misuse of the instrument.
 CAUTION	A caution alerts the user to possible injury or problems with the instrument associated with its use or misuse such as instrument malfunction, instrument failure, damage to the instrument, or damage to other property.

Icons in this Manual

Icon	Description
	Gives additional information and alternative operation methods.
	Indicates related pages in this or other manuals which give more details.

Text Conventions in this Manual

Style	Description
xxx key	Operating panel key
[xxx] tab	Window tab key
[xxx] window	Window name appearing on tab
[xxx]	Window touch key
"xxx"	Window message

Safety Standards

Safety Standard Classification of the CSM-1901 Bedside Monitor

CSM-1901 Op No. 01A

Type of protection against electrical shock:
CLASS I EQUIPMENT

Degree of protection against electrical shock:
Defibrillation-proof type CF applied part

Degree of protection against harmful ingress of water:
IPX1 (protected against vertically falling water drops)
Only when the system configuration is CU-191R or CU-192R, VL-190P and DM-190P

Method of cleaning and disinfecting or sterilization:
Equipment NOT suitable for sterilization

Degree of safety of application in the presence of FLAMMABLE ANAESTHETIC MIXTURE WITH AIR, OR WITH OXYGEN OR NITROUS OXIDE:
Equipment not suitable for use in the presence of FLAMMABLE ANAESTHETIC MIXTURE WITH AIR, OR WITH OXYGEN OR NITROUS OXIDE

Mode of operation:
CONTINUOUS OPERATION

ME EQUIPMENT type:
STATIONARY type

For the following units, refer to the manual for each unit.

- BSM-1733, BSM-1753, BSM-1773 bedside monitor
- GF-210R multigas unit
- AE-918P neuro unit

CSM-1901 Op No. 01K

Type of protection against electrical shock:
CLASS I EQUIPMENT

Degree of protection against electrical shock:
BF applied part: TOF

Defibrillation-proof type CF applied part:
ECG, impedance respiration, thermistor respiration, IBP, temperature, SpO₂, SpO₂-2, CO₂, O₂, BIS, NIBP

CF applied part: CO

Degree of protection against harmful ingress of water:
IPX1 (protected against vertically falling water drops)
Only when the system configuration is CU-191R or CU-192R, VL-190P and DM-190P

Method of cleaning and disinfecting or sterilization:
Equipment NOT suitable for sterilization

Degree of safety of application in the presence of FLAMMABLE ANAESTHETIC MIXTURE WITH AIR, OR WITH OXYGEN OR NITROUS OXIDE:
Equipment not suitable for use in the presence of FLAMMABLE ANAESTHETIC MIXTURE WITH AIR, OR WITH OXYGEN OR NITROUS OXIDE

Mode of operation:
CONTINUOUS OPERATION

ME EQUIPMENT type:
STATIONARY type

For the following units, refer to the manual for each unit.

- BSM-1733, BSM-1753, BSM-1763, BSM-1773 bedside monitor
- GF-110PA, GF-210R, GF-310R multigas unit
- GF-120PA, GF-220R, GF-320R multigas/flow unit
- AE-918P, AE-920P neuro unit
- AF-101P NMT module
- AP-170P hemodynamic unit

1

Bedside Monitor Overview

General.....	1-2
Indications for Use.....	1-3
Basic Configuration	1-4
Notes for Product Use	1-5
Administrator Settings	1-5
Basic Operation.....	1-6
Operation Methods.....	1-6
Using the Touch Screen and Operation Keys.....	1-6
Using the Remote Control.....	1-6
Using the Keyboard and Mouse.....	1-6
Multiple Displays.....	1-7
Precautions for Using Multiple Displays	1-7
Window Display.....	1-7
Volume Setting.....	1-7
Menu Window.....	1-8
Home Screen.....	1-9
Symbols.....	1-10
Transport Package	1-11

General

The CSM-1901 bedside monitor has a color display and is for one patient and is to be installed near the patient. With the basic configuration of the system, ECG, NIBP, temperature, SpO₂, respiration in impedance or thermistor method, CO₂, O₂ of all hospital patients can be monitored and alarms are generated.¹ Apnea and arrhythmia can also be monitored.

¹ Essential performance of the bedside monitor

Adults, children and neonates can be monitored using this bedside monitor.

You can use other units and options with the bedside monitor to add other parameters so the bedside monitor can be used in a wide range of sites such as operating rooms and intensive care units (ICU).

You can also connect the bedside monitor to a network to communicate with a central monitor and other bedside monitors.

CSM-1901 bedside monitor has two models, Op No. 01A and Op No. 01K. The option number is printed on the name plate of the unit. For simplicity, the option number is omitted in this manual. There is no difference in operation between two models unless otherwise specified.



- Descriptions in this manual are based on a basic configuration with the following units.
 - VL-190P LCD unit
 - CU-191R core unit
 - JA-694PA data acquisition unit
 - AY-673P input unit
- The window examples in the descriptions in this manual are factory default settings (OR mode).
- The following units are IEC 60601-1:1988+Amendment 1:1991+ Amendment 2:1995 compliant.
 - AG-920RK multigas unit
 - TG-950P CO₂ sensor kit
 - VL-611R (ET2400L-8CJM-1-BG-NKD-G) LCD unit

WARNING

Do not use the same bedside monitor for more than one patient at the same time. Do not connect different sensors from different patients to the same bedside monitor.

WARNING

Do not diagnose a patient based only on data acquired by the bedside monitor. Overall judgement must be performed by a physician who understands the features, limitations and characteristics of the bedside monitor and by reading the biomedical signals acquired by other instruments.

⚠ CAUTION

Only use Nihon Kohden specified options and consumables such as electrodes, sensors, probes and transducers. Otherwise, the maximum performance from the bedside monitor cannot be guaranteed.

- NOTE**
- This bedside monitor must be used by qualified medical personnel with full knowledge of its functions and operations.
 - Changing of settings which affect the operation of the bedside monitor, including location, requires supervision or confirmation from the administrator. Improper changes may result in monitoring errors or overlooked alarms.
 - Upgrade the core unit and each optional unit to the Nihon Kohden recommended software version. If more than one CSM-1901 bedside monitor is used in the same facility, make sure the bedside monitors have the same software version. If CSM-1901 bedside monitors with different software versions are used together, correct system operation cannot be guaranteed.

Indications for Use

The CSM-1901 bedside monitor is intended to monitor, display and record physiological data to provide cardiac and vital signs monitoring within a medical facility. The device is intended to produce a visual record of the electrical signal produced by the heart and monitor the electrocardiogram to generate visible and/or audible alarms when an arrhythmia exists. The device is also intended to monitor heart rate, pulse rate, blood oxygen saturation (SpO₂), non-invasive blood pressure (NIBP), invasive blood pressure (IBP), body temperature, BIS, cardiac output (CO), oxygen concentration (FiO₂), carbon dioxide concentration (CO₂), EtCO₂, respiratory rate, inspired and expired anesthetic agents and anesthetic gases including N₂O, halothane, isoflurane, enflurane, sevoflurane and desflurane and TOF. The device also displays patient data from external devices such as ventilators, TOF monitors, CCO/SvO₂ monitors, and EEG measuring unit.

The device may generate and audible and/or visual alarm when a measured rate falls outside preset limits.

The device will be available for use by trained medical personnel within a medical facility on all patient populations, including adult, neonate, infant, child, and adolescent subgroups.

Basic Configuration

With the following basic system configuration, the ECG, respiration using the impedance or thermistor method, NIBP, SpO₂, temperature, IBP, CO₂, cardiac output, O₂, BIS and TOF of hospital patients can be monitored.

The bedside monitor can display measurement data when the external instruments such as a multigas unit or ventilator are connected to the bedside monitor.

NOTE: The system configuration depends on the country. Contact your Nihon Kohden representative for the available options for your country.

- CU-191R, CU-192R core unit
- VL-190P LCD unit
- YL-920P alarm indicator
- AY-631P, AY-633P, AY-651P, AY-653P, AY-660P¹, AY-661P¹, AY-663P¹, AY-671P, AY-673P input unit

Mdel	MULTI Sockets	SpO ₂ Measurement
AY-631P	1	Masimo
AY-633P	3	
AY-651P	1	Nellcor
AY-653P	3	
AY-660P ¹	1	Nihon Kohden
AY-661P ¹	1	
AY-663P ¹	3	
AY-671P	1	
AY-673P	3	

- BSM-1700 series bedside monitor

BSM-1700 series bedside monitor is used as an input unit.

Mdel	SpO ₂ Measurement
BSM-1733	Masimo
BSM-1753	Nellcor
BSM-1763 ¹	Nihon Kohden
BSM-1773	

- AA-672P, AA-674P smart expansion unit
- JA-690PA, JA-694PA data acquisition unit
- AA-910P multi amp unit
- JA-920P interface unit
- WS-960P recorder unit
- RY-910PA remote controller

- SB-920P battery pack

¹ The AY-660P, AY-661P and AY-663P input units and BSM-1763 bedside monitor are not available in North America.

Notes for Product Use

- Antivirus software is installed on the bedside monitor at the factory to prevent virus infection from the hospital network or a USB flash drive.
- The exterior of the bedside monitor may become hot, but this is not an abnormality.
- Do not format a USB flash drive which contains the bedside monitor data to the file system of other medical equipment or IT equipment. The USB flash drive data will be lost.

Administrator Settings

The administrator of the bedside monitor can change the operating environment settings according to the intended use, measurement and operation.

There are two windows for changing the settings.

System Configuration

The administrator can change the site and network settings.

For details, refer to Section 2 of the Administrator's Guide.



Administrator's Guide:
Refer to Section 2 "System Configuration"

System Setup

The administrator can change the bed ID and alarm master settings.

For details, refer to Section 3 of the Administrator's Guide.



Administrator's Guide:
Refer to Section 3 "System Setup"

Basic Operation

Operation Methods

The bedside monitor can be operated by the following methods. These methods can be used together.

- Touch screen
- Operation keys on components
- Remote control
- Mouse
- Keyboard

This manual mainly describes operations using the touch screen.

Using the Touch Screen and Operation Keys

You can operate the monitor by touching the keys on the screen or pressing the operation keys.

By touching the keys on the screen directly, you can intuitively operate the bedside monitor.

By pressing the operation keys, you can display the windows or perform the key operations.

For details on operation keys, refer to “Panel Description”.

Touch the keys on the screen to change the settings, display or close the window or scroll the bar.

NOTE: Only use your fingers to operate keys on the touch screen. Using a sharp object may damage the bedside monitor.

Using the Remote Control

Use the RY-910PA remote controller to operate the bedside monitor from a distance.

Use the selection knob on the remote controller to move the pointer on the screen and press the [Enter] key to select the item.

- NOTE**
- If two bedside monitors are near each other, confirm that the remote control operates only the intended bedside monitor. If the remote control operates a different bedside monitor, check the channel setting.
 - Watch the monitor screen and check the operation when using the remote control.
Store the remote control carefully to prevent accidental button operation.

Using the Keyboard and Mouse

A keyboard and mouse can be connected.

Use the keyboard for entering letters, symbols and numbers. Use the mouse to move the pointer on the window, and click the left button to select and register settings.

Multiple Displays

Monitoring with multiple displays is possible when using the bedside monitor.

The window layout and parameters can be set individually for each display, according to the application or installation location.

NOTE: Do not turn the display power off during monitoring.

Precautions for Using Multiple Displays

Window Display

When using the bedside monitor with multiple displays, the same window cannot be displayed at the same time on multiple displays.

If you touch a second display while an operation is still pending on the first display, a message appears on the second display. Touch [Yes] or [No] to close one of the windows.

NOTE: If a window was closed by an operation on the other display while a setting change was in progress, check the setting again.

Volume Setting

When using the bedside monitor with multiple displays, the volume cannot be set separately on each display.



If the volume setting is changed on one display, it is also changed on the other.


Menu Window

The [Menu] window gives one-touch access to all the windows that can be displayed on the bedside monitor.

The keys are located according to the functions on the window. Touch a key to display the corresponding window.

There are four ways to display the [Menu] window:

- Press the **Menu key**
- Touch the [Menu/Home] function key
 -  Available when “Menu/Home” is assigned to a function key in System Setup.
Administrator’s Guide: Refer to “Setting Function Keys” in Section 3
- Press the **MENU/HOME key** on the RY-910PA remote controller
 -  Press the MENU/HOME key on the remote control (option) while a window other than the home screen is displayed to return to the home screen.
Press the MENU/HOME key again to display the [Menu] window.
- Press the **Menu key** on the VL-190P LCD unit

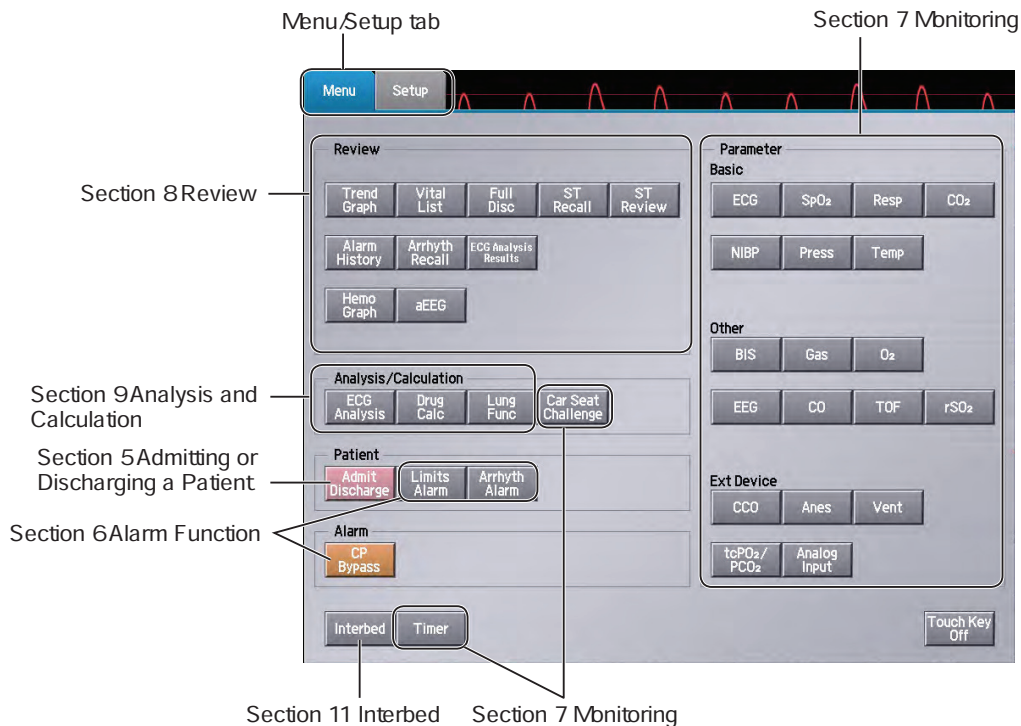

Flow/Paw, esCCO and TOF using an AF-101P NMT module are not available in North America.

Car Seat Challenge is only available in North America and only when the [Site] setting is [NICU].


StO₂ is displayed instead of rSO₂ when an IF-963P communication cable is used. The IF-963P communication cable is only available in North America.

Home Screen

The home screen displays the patient monitoring information. The home screen is the main window of the bedside monitor.

There are five ways to display the home screen. Whichever method is used, the home screen appears immediately.

- Touch the waveform display area
- Touch the [Home/Menu] function key
 - Available when [Menu/Home] is assigned to a function key in System Setup.
Administrator's Guide: Refer to "Setting Function Keys" in Section 3
- Press the **MENU/HOME** key on the RY-910PA remote controller
- Press the **Home** key on the VL-190P LCD unit
- Press the **Home** key on the JA-690PA or JA-694PA data acquisition unit

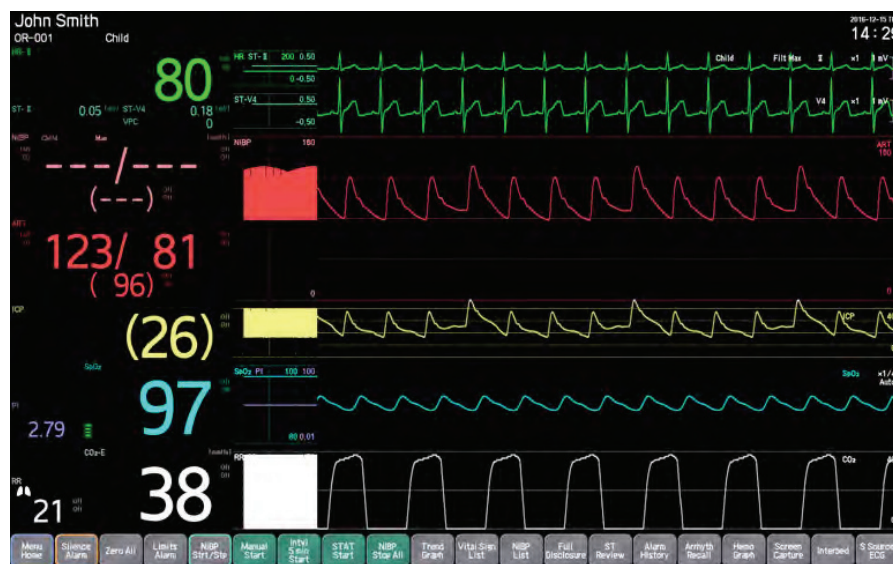

Home screen

Symbols

The following symbols are used with the bedside monitor. The names and descriptions of each symbol are as shown in the table below.

For the following units, refer to the manual for each unit.

- GF-110PA¹, GF-210R, GF-310R¹ multigas unit
- GF-120PA¹, GF-220R¹, GF-320R¹ multigas/flow unit
- AE-918P, AE-920P¹ neuro unit
- AF-101P NMT module¹
- AP-170P hemodynamic unit¹
- BSM-1733, BSM-1753, BSM-1763¹, BSM-1773 bedside monitor

¹ The GF-110PA and GF-310R multigas units, GF-120PA, GF-220R and GF-320R multigas/flow units, AF-101P NMT module, AP-170P hemodynamic unit, AE-920P neuro unit and BSM-1763 bedside monitor are not available in North America.

Symbol	Description
	“On” only for a part of instrument
	“Off” only for a part of instrument
	Standby
	Alternating current
	Direct current
	Equipotential terminal
	Recorder socket/Recorder
	Input/output terminal (Multi-link socket, EJA unit socket, DAU unit socket, ViSiBLE unit socket and Serial socket)
	Output terminal (Front Panel: ECG/BP OUT socket, Unit connection socket)
	Input terminal
	Sound
	Silence alarms key
	Out of paper
	Attention, see instructions for use/ Caution
	Follow instructions for use
	Background color: Blue
	USB socket

Symbol	Description
	Display 1 terminal
	Display 2 terminal
	Display 3 terminal
	Protective earth
	Media access lamp
	Nihon Kohden network terminal
	Hospital network terminal
	Defibrillation-proof type CF applied part
	BIS READY label (QE-910P BIS processor/BISx processor can be connected)
	Alarm indicator
	Direction for inserting battery
	NIBP interval
	NIBP start
	NIBP stop
	Alarm silence
	Audio off

Symbol	Description
	Alarms paused
	All alarms off/Vital sign alarm limit off
	Non-paced
	QRS/pulse sync mark
	Respiration sync mark
	MR unsafe
	NIBP
	Recycle
	The CE mark is a protected conformity mark of the European Community.
	The CE mark is a protected conformity mark of the European Community. The four digits after the CE mark indicate the identification number of the Notified Body involved in assessing the product's conformity as a medical device.
	Products marked with this symbol comply with the European WEEE directive 2012/19/EU and require separate waste collection. For Nihon Kohden products marked with this symbol, contact your Nihon Kohden representative for disposal.

Symbol	Description
	IBP all zero
	PC starting
	Adjust setting/Scroll data
	Left end/Right end
	Serial number
	Date of manufacture
	Recording settings
	Printer
	Car seat challenge time
	HiQ-View
	CAUTION: United States law restricts this product to sale by or on the order of a physician.
	Products marked with this symbol comply with an environmental protection use period of 10 years according to the SJ/T11364 "Marking for the Restricted Use of Hazardous Substances in Electronic and Electrical Products" of the People's Republic of China Electronic Industry Standard.

Transport Package

Symbol	Description
	This way up
	Fragile

Symbol	Description
	Keep away from rain
	Stacking limit by number ("n" is the limiting number)

7

Monitoring

7

7-1

7-2

7-3

Monitoring Overview.....	7-0-2
Home Screen.....	7-0-4
Displaying the Home Screen.....	7-0-4
Example of Home Screen	7-0-4
Home Screen Settings	7-0-5
Window Display.....	7-0-5
Recording.....	7-0-5
Display Settings in System Setup	7-0-5
Short Trend.....	7-0-6
OCRG.....	7-0-7
IBP Display Types	7-0-8
Freezing Waveforms	7-0-9
Unfreezing Waveforms.....	7-0-9
Changing to Another Window.....	7-0-9
Home Screen Layout.....	7-0-10
Timer Function.....	7-0-10
Showing Timers	7-0-10
Using the Timer.....	7-0-11
Setting the Timer.....	7-0-11
Browser Function	7-0-11
Using Sleep Mode.....	7-0-12
Exiting Sleep Mode	7-0-13
Disabling the Touch Keys	7-0-13
Restoring Touch Key Operation.....	7-0-14
Using the Car Seat Challenge Function.....	7-0-14
Basic Parameters	7-1
Other Parameters.....	7-2
External Instrument Parameters.....	7-3

Monitoring Overview

When monitoring starts, waveforms and measurement data appear on the home screen and the corresponding parameter windows.

To display a parameter setting window, touch [Menu] on the home screen ► parameter key. The tabs or keys show the setting items.

Basic parameters

- [ECG] window
- [SpO₂] window
- [Resp] window
- [CO₂] window
- [NIBP] window
- [Press] window
- [Temp] window

Other parameters

- [BIS] window
- [Gas] window
- [Flow/Paw] window¹
- [O₂] window
- [EEG] window
- [CO] window
- [TOF] window
- [rSO₂]/[StO₂]³ window
- [Incu] window⁴

External instrument parameters

- [CCO] window
- [esCCO] window²
- [Anes] window
- [Vent] window
- [tcPO₂/PCO₂] window
- [Analog] window

¹ Flow/Paw is not available in North America.

² esCCO is only available with the QP-192P esCCO program or QP-193P hemodynamics review program. The QP-192P esCCO program and QP-193P hemodynamics review program are not available in North America.

³ StO₂ is displayed instead of rSO₂ when an IF-963P communication cable is used. The IF-963P communication cable is only available in North America.

⁴ Incubator monitoring is only available in North America. The [Incu] window is only displayed when [Site] is set to [ICU] or [NICU] in System Configuration.



Administrator's Guide: Refer to "Site Settings" in Section 2

This section describes monitoring for the basic parameters and other parameters listed above.

WARNING

Do not diagnose a patient based only on data acquired by the bedside monitor. Overall judgement must be performed by a physician who understands the features, limitations and characteristics of the bedside monitor and by reading the biomedical signals acquired by other instruments.

⚠ WARNING

After attaching electrodes, probes and sensors on the patient and connecting cables to the bedside monitor, check that there is no error messages and the waveforms and numeric data are appropriately displayed on the screen. If there is an error message, or waveform or numeric data is not appropriate, check the electrodes, probes and sensor attachment, patient condition and settings on the bedside monitor and remove the cause.

7

⚠ WARNING

A physician must be within the range where he/she can hear the alarm sound of the bedside monitor while monitoring a patient on the bedside monitor. If the physician cannot hear the alarm sound, critical changes on the patient may be overlooked.

Home Screen

The home screen displays monitoring information such as measurement data and waveforms.

The window layout is selected automatically according to the measured parameters. The window display type and value display positions can be set in the [Screen Layout Select] window of System Setup.



Administrator's Guide:

Refer to "Changing the Master Layout" in Section 3


Displaying the Home Screen

To close an open window and return to the home screen, press the **Home key**.

The display can also be returned to the home screen with the following operations.

- Touch the waveform/short trend display area
- Touch the [Menu Home] function key (only when set as a function key)
- Press the **MENU/HOME key** on the remote control
- Press the **Home key** on the VL-190P LCD unit

Example of Home Screen

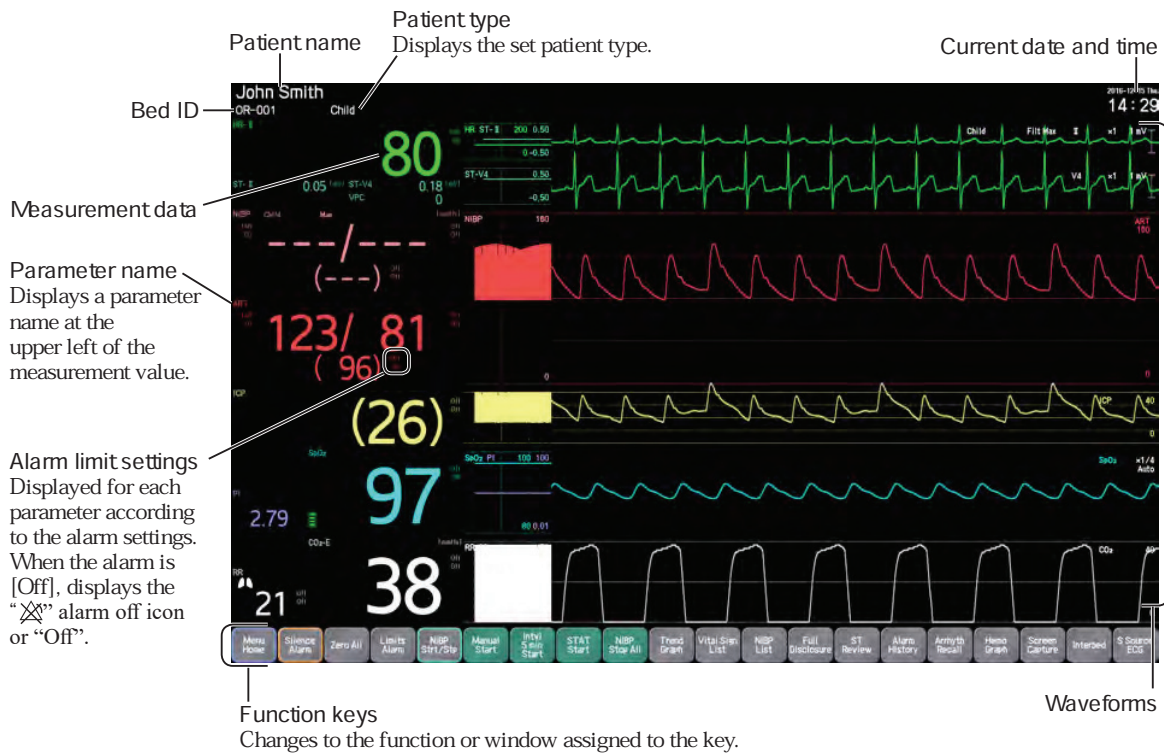

The following windows can be displayed directly from the home screen.

- [Admit Discharge] window: Touch a patient name
- Window assigned to a function key: Touch the function key
- Parameter windows: Touch a parameter display area
- Review windows: Touch a bottom or side panel

Home Screen Settings

The home screen displays the monitoring information.

Window Display

- The layout of the home screen can be set.
([Menu] window ► [Setup] tab ► [Screen Select])
- The sensitivity and scale of waveforms can be set.
(Corresponding parameter window)
- The sweep speed of waveforms can be changed.
([Menu] window ► [Setup] ► [Settings] ► [Sound/Display] tab ► [Wave] window)
- The pulse rate can be turned on or off.
([Menu] window ► [ECG], [SpO₂], [Press] or [NIBP] ► “PR Display” in [Detailed Settings] window)
- Only one method can be used for detecting the respiration rate (RR). When more than one respiration parameter is measured, only the values for the highest priority method shown below are displayed.

Respiration rate (RR) priority

Anesthetic gas > Flow > CO₂ > Thermistor respiration > Impedance respiration

Recording

- When a WS-960P recorder unit (option) is connected to the bedside monitor, up to 3 waveforms set in the [Setup] window can be recorded. When the bedside monitor is connected to a network, up to 10 waveforms set in the [Setup] window can be printed on the network printer.
- When the bedside monitor is connected to a network, recording can be performed with the central monitor recorder (remote recording).
- When the bedside monitor is connected to a network which has a network printer, a captured screen can be printed with a network printer.
([Settings] of the System Configuration ► [Other] window ► Capture File Redirection)



Administrator's Guide: Refer to Section 2 “System Configuration”

Display Settings in System Setup

The following display settings can be changed in System Setup.



Administrator's Guide: Refer to Section 3 “System Setup”

- The window layout can be customized.
(System Setup ► [Screen Layout Select] ► [Edit Layout] window)
- The date format can be set.
(System Setup ► [Settings] ► [Sound/Display] tab ► [Display] window ► [Date Format])

- The waveform display mode (moving, fixed) can be changed.
(System Setup ► [Settings] ► [Sound/Display] tab ► [Waveform] window ► “Wave Display”)
- Trace 1, 2 or 3, or all leads, can be displayed for the ECG.
(System Setup ► [Screen Layout Select] ► [Edit Layout] tab ► Numeric area, Short Trend area or Waveform area ► [Parameter] window ► “Trace”)
- The short trend can be hidden, or changed to the OCRG display.
(System Setup ► [Screen Layout Select] ► [Edit Layout] tab ► [Change Layout] window ► “Short Trend”)
- The IBP waveform and short trend display type ([Common], [Separate], [Dual]) can be changed.
(System Setup ► [Screen Layout Select] ► [Edit Layout] tab ► Numeric area, Short Trend area or Waveform area ► [Parameter] window ► “Pressure Scale”)
- The ICP scale display type ([Common], [Separate]) can be changed.
(System Setup ► [Screen Layout Select] ► [Edit Layout] tab ► Numeric area, Short Trend area or Waveform area ► [Parameter] window ► “ICP Scale”)
- Anesthesia gas and CO₂ measurement values in the numeric display area can be changed by swiping on them.
(System Setup ► [Param] ► [Gas] tab ► [Display Numeric Priority] tab)
- Parameter window frames can be displayed for the value display areas. The parameter window frames are displayed around the value display areas that separate each parameter.
(System Setup ► [Settings] ► [Sound/Display] tab ► [Values] tab ► [Border Around Numeric Frame])
- Up to five timers and current time can be displayed on the home screen.
(System Setup ► [Layout] ► [Edit Layout] tab ► [Change Layout] window ► [Timer])

Short Trend

The short trend is a trend for the past 30 minutes or less. You can display a short trend on the home screen.

The short trend can be set to off or changed to the OCRG display in System Setup.

Showing or hiding of the short trend can be changed by touching the [Short Trend On/Off] function key when it is set as a function key.

Short trend description

- The scale for each parameter is set in the corresponding parameter window.
- The display method for the IBP short trend depends on the setting for the BP waveform display type (Separate, Common, Dual).
- The short trend and scale values are displayed in the display color of the corresponding parameter. The display colors can be set in the System Setup.
- The length of time for the short trend display can be set from 0 to 30 minutes by touching the screen.
- When NIBP and arterial IBP (ART, ART2, RAD, DORS, AO, FEM, UA, LVP, P1 to P8) are measured at the same time, the short trend of the NIBP measured values is superimposed on the short trend of the arterial IBP measured values.

OCRG

The short trend display on the home screen can be changed to the OCRG display.

The OCRG (oxycardio respirogram) is a trend graph that shows the heart rate, respiration, and oxygenation condition (SpO₂) correlated to changes in these parameters on the same time axis during neonate monitoring.

The OCRG display area includes the following data.


- Heart rate trend graph
- SpO₂ trend graph
- Compressed respiration waveform
- tcPO₂/tcPCO₂ trend graph (only when set in “OCRG” of System Setup ► [Settings] ► [Sound/Display] tab ► [Display] window)
- EEG trend graph (only when set in “OCRG” of System Setup ► [Settings] ► [Sound/Display] tab ► [Display] window)

7

Change to the OCRG display in System Setup ► [Screen Layout Select] ► [Edit Layout] ► [Change Layout] window.

Set the OCRG display interval with “OCRG Display Interval” in System Setup ► [Sound/Display] ► [Display] window.


- Respiration rate detection can only be performed with the impedance method or the thermistor method. If both are used for measuring at the same time, detection is performed with the impedance method, and the corresponding measured values are displayed.
- The displayed OCRG can be recorded with the WS-960P recorder unit (option).

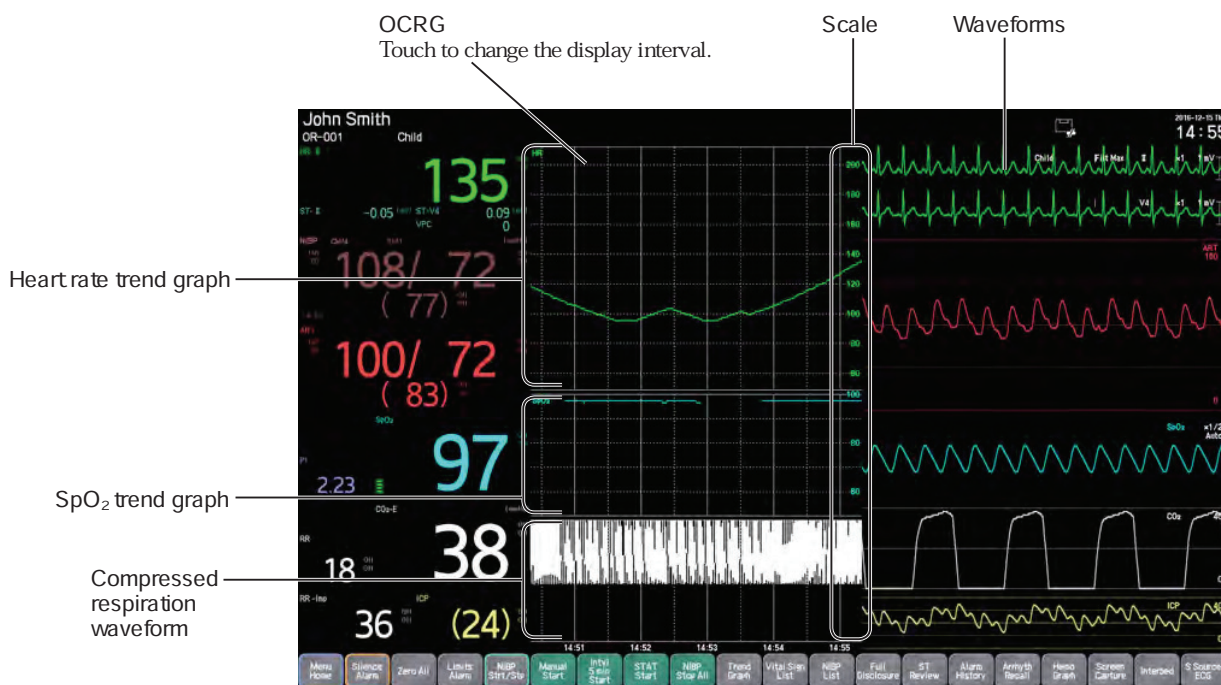

IBP Display Types

You can display the IBP waveforms and short trends in separate display for each channel, superimposed in common display area or with high pressure arterial pressures superimposed in one common display area and low pressures superimposed in another common display area (Common, Separate, Dual).

"Common" BP display

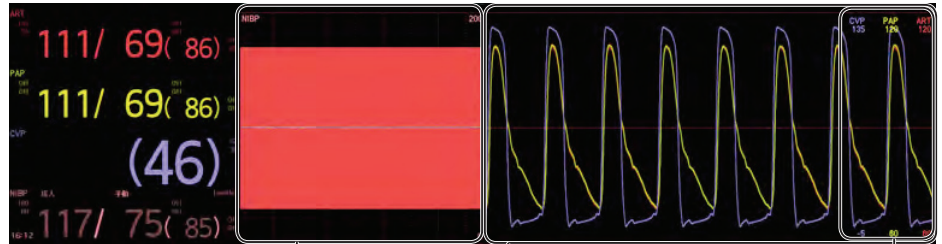


BP waveform short trends:
Sys (Max) and Dia (Min) for highest priority BP channel, Mean for the other BP channels

BP waveforms
(Up to 8 waveforms)

BP waveform scales

"Separate" BP display

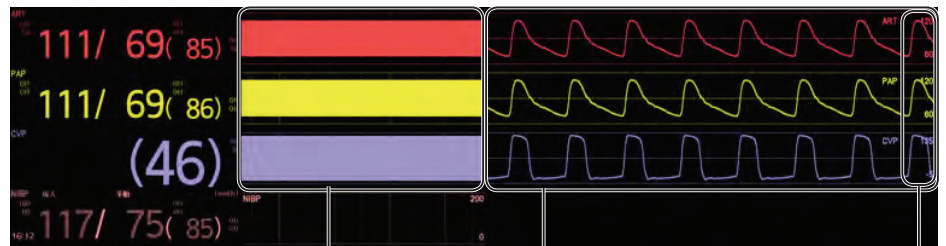


BP waveform short trends:
Sys and Dia

BP waveforms
(Up to 8 waveforms)

BP waveform scales

"Dual" BP display

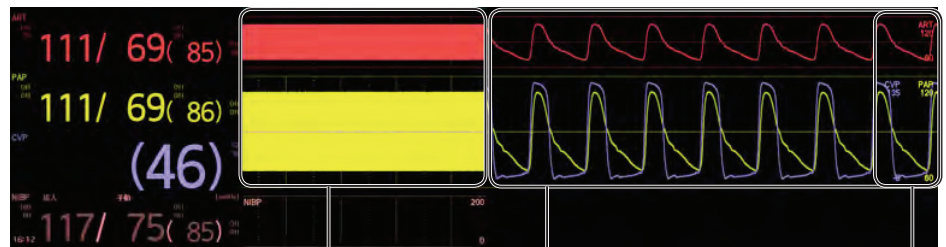

BP waveform short trends:
Sys and Dia

BP waveforms
(Up to 8 waveforms)

BP waveform scales

Freezing Waveforms

To freeze all the waveforms displayed on the home screen, touch the Freeze function key.

NOTE: Unfreeze to continue monitoring.

Unfreezing Waveforms

The waveforms can be unfrozen in the following ways.

- Touch [Freeze] again
- Touch the touch screen
- Press any key

Touch a key below for the corresponding function or window.

Silence Alarms key: If the alarm status is present at the moment of silencing, enters the alarm silenced status at the same time.

Start/Stop key: Starts manual measurement at the same time.

Menu key: Displays the [Menu] window.

- Three minutes elapse (unfrozen automatically)
- When the [Interbed] window is displayed automatically



[Interbed] window display:

“Changing the Interbed Alarm Settings” in Section 11

Changing to Another Window

The [Menu] window has keys for all windows on the bedside monitor. Some windows such as parameter windows or [Admit Discharge] window can also be accessed from the home screen.

Using the [Menu] window to access another window

- 1 Press the **Menu key** (VL-190P LCD unit, JA-690PA or JA-694PA data acquisition unit) to display the [Menu] window.

The [Menu] window can also be displayed by touching the [Menu Home] function key when it is set as a function key.

- 2 Touch the key of the desired window.

The review window which is set as a function key can also be displayed by touching the review function key when it is set as a function key.

Changing directly from the home screen

The following windows can be accessed from the home screen.

- Touch a parameter numeric value: Displays the corresponding parameter window
- Touch a patient name: Displays the [Admit Discharge] window
- Touch a function key: Displays the window assigned to the function key
- Touch a bottom panel: Displays the corresponding review window
- Touch a side panel: Displays the corresponding review window

Home Screen Layout

The home screen of the bedside monitor can be customized.

Customized window layouts can be saved and used later.



Window layout settings:

Refer to “Changing the Master Layout” in Section 3 of the Administrator’s Guide

CAUTION

When the screen layout is set to [Auto], not all parameters can be displayed on the screen when too many parameters are monitored. When the screen layout is set to [Fixed], only the assigned parameters are displayed on the screen regardless of whether the parameter is measured or not. At the start of monitoring, check which parameters are displayed and which parameters are not displayed on the screen. When an alarm occurs on the measured parameter, the alarm is displayed regardless of whether the parameter is displayed or not displayed.

Timer Function

Up to five timers can be displayed on the home screen. Use the timer to time measurement. You can set each timer individually on the [Timer] window ► [Monitor Setting] in System Setup.



Refer to “Timer” in Section 3 of the Administrator’s Guide

Showing Timers

You can display the timers by either of the following methods.

- Display all timers by touching [Timer] on the Menu window


This displays all timers on the home screen.

You can also display all the timers with the [Timer] function key on the remote control if it is assigned as a function key.

- Display one or more individual timers by touching the [Timer] function key (when assigned as a function key)

Select one or more of the five timers ([Timer 1] to [Timer 5]) to display the timer on the home screen.

Timer

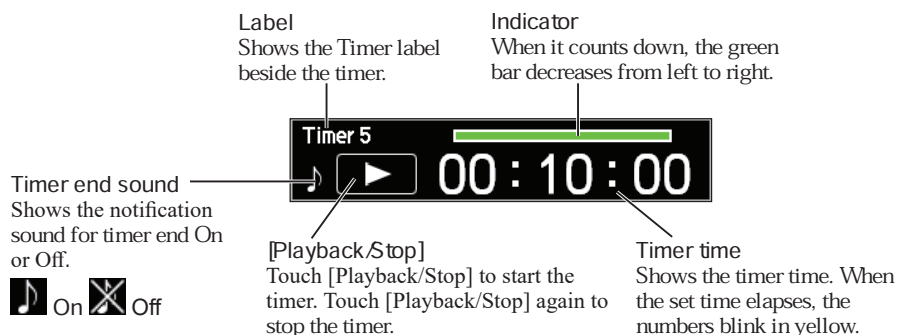

7

Changing the Timer Layout

Swipe to the right on the timer to display the [Row Layout] key and [Vertical Layout] key.

- [Row Layout]: Arrange all the timers on the screen horizontally from left to right.
- [Vertical Layout]: Arrange all the timers on the screen vertically from top to bottom.

Using the Timer

Start/Stop Counting

Touch [Playback/Stop] on the timer to start counting up or counting down. To stop counting, touch [Playback/Stop] again.

Retting the Timer

To reset the timer, swipe to the left on the timer and touch the [Reset] key.

Hiding a Timer

To remove a timer from the screen, swipe to the left on the timer and touch the [Delete] key.

Setting the Timer

- 1 Touch the time on the timer.
- 2 Set each item on the Timer setting window and touch [Set] to apply the changes. For the settings, refer to the Administrator's Guide.



Refer to "Timer" in Section 3 of the Administrator's Guide

Browser Function

To display the browser on the window, touch the [Browser] function key. The browser display area is fixed.

NOTE: Do not perform unspecified update of the browser. The bedside monitor might not function properly.

Using Sleep Mode

Sleep mode can be used when all the following conditions are satisfied.

- [Site] in System Configuration is set to “ICU”, “NICU”, “Ward” or “ER”.
- “Alarm Off Type” is set to “Alarms Suspended” in System Setup ► [Settings] ► [Alarm] tab ► [Paused/Silence] window.
- The bedside monitor is connected to a network, and the patient being monitored by the bedside monitor is also being monitored at the same time from another monitor, such as a telemetry system or central monitor.



When more than one display is used, sleep mode can be used for each individual display.

When [Sleep] is touched on the [Menu] window and then [Yes] is touched, the bedside monitor operates as follows.

- The screen dims.
- The sync sound is no longer emitted.
- The alarm indicator turns off. (No longer blinks synchronized with the sync sound.)

WARNING

When using sleep function, monitor the patient on the central monitor or telemetry system. Otherwise, the bedside monitor alarm may be overlooked. When [Exit Sleep Mode on Crisis Alarm] check box on the ALARM page of the System Setup window is [Off], bedside monitor alarms and sync sound appear on the central monitor but do not appear on the bedside monitor during sleep mode.

Exiting Sleep Mode

Sleep mode is exited when any of the following occurs. When sleep mode is exited, the home screen returns.

- Touching the touch screen
- Pressing any key

Touch a key below for the corresponding function or window.

Silence Alarms key: If the alarm status is present at the moment of silencing, enters the alarm silenced status at the same time.

Start/Stop key: Starts manual measurement at the same time.

Press all zero key: Zero balance of all blood pressure lines exposed to air is performed.

Interval key: Changes the NIBP measurement interval.

Menu key: Displays the [Menu] window.

Home key: Displays the home screen.

7

- When the auto exit time elapsed

Set the auto exit time for the sleep mode in the administrator window of the alarm settings.

- A crisis alarm occurs (automatically exits the sleep mode)

When “Exit Sleep Mode on Crisis Alarm” is set to [On] in the administrator window of the alarm settings, the sleep mode is exited when a crisis alarm occurs.

- When disconnected from the network
- When admitting or discharging a patient from the central monitor
- When suspending monitoring from the central monitor
- When a parameter is added
- When the input unit in the JA-690PA or JA-694PA data acquisition unit is removed

Disabling the Touch Keys

The touch keys are disabled when you touch the [Touch Key Off] on the [Menu] window. This is useful when cleaning the touch screen.

- 1 Touch the [Touch Key Off] on the [Menu] window to display the Touch Key Off window.

The Touch Key Off window can also be displayed by touching [Touch Key Off] function key (Only when set as a function key.)

- 2 Touch [Yes] to disable the touch keys.



- To cancel disabling touch keys, touch [No]. The window returns to the home screen.
- If the bedside monitor has more than one display, the touch keys on all displays are disabled.

Restoring Touch Key Operation

Touch key operation is restored when any of the following occurs.

- Pressing a key
(JA-690PA or JA-694PA data acquisition unit, BSM-1700 series bedside monitor, RY-910PA remote controller)

Touch a key below for the corresponding function or window.

Silence Alarms key: If the alarm status is present at the moment of silencing, enters the alarm silenced status at the same time.

Start/Stop key: Starts manual measurement at the same time.

Menu key: Displays the [Menu] window.

Home key: Displays the home screen.

Record key: Starts manual recording at the same time.

- Pressing the Power switch, Check key or Service key
(CU-191R, CU-192R core unit, VL-190P LCD unit)
- Three minutes elapse (restored automatically)

Using the Car Seat Challenge Function

Car Seat Challenge is only available in North America.

The American Academy of Pediatrics recommends a car seat challenge for all neonates born before 37 weeks gestation to ensure that the neonate is able to sit in a car seat safely without any episodes of oxygen desaturation apnea or bradycardia.

The Car Seat Challenge window measures a neonate's lower heart rate limit, lower SpO₂ limit and apnea time for a set duration. This is only available when the [Site] setting is [NICU].

- 1 On the [Menu] window, touch the [Car Seat Challenge] key. The Test window is displayed.
- 2 Set the test duration and touch the [Start] key. The [Car Seat Challenge] window appears.

To stop the test, touch the [Stop] key.

After the set duration, the [Results] window appears.

You can set display of the countdown timer window for the car seat challenge on or off.



Administrator's Guide:

"Setting Display of the Cart Seat Challenge Time Window On or Off"
in Section 3

- 3 Touch the [Report] key to print the report. The [Report Settings] window is displayed.
- 4 The name of the person who performed the test and any comments can be entered.
 - 1) Touch the Name column and enter the name (up to 16 characters). Touch the Comments column to enter comments with the displayed keyboard.
 - 2) Touch the [Set] key. The window closes.

- 5 Touch the [Print] key on the [Report Settings] window. Change the time range to print if necessary.

The test results are printed on the network printer.

Index

Symbols and Numbers

1 Hz Mode	7-2-47
12 Lead Results	
[Analysis Report] Window	8-38
[Analyzed Waves] Window	8-36
[Averaged Waves] Window	8-37
12-Lead Waveforms Recording	10-3-7

A

AA-910P Multi Amp Unit	2-15
Cleaning and Disinfection	14-15
Adjusting Zero Balance [BP]	7-1-79
Adjusting Zero Balance (When Using TG-950P, TG-970P, TG-980P) [CO ₂]	7-1-57
Admitting a Patient	5-2
Entering Patient Information	5-2
AE-918P Neuro Unit	
Cleaning and Disinfection	14-25
AE-920P Neuro Unit	
Cleaning and Disinfection	14-25
AF-101P NMF Module	
Cleaning and Disinfection	14-25
AG-400RK CO ₂ Unit	
Cleaning and Disinfecting Connection Cables	14-25
Alarm Auto Recording	10-0-7
Alarm History	8-27
Alarm History Recording	10-2-19
Alarm Recording	6-18, 10-1-4
Alarm Sound Volume	6-18
Alarm Types and Levels	6-3
Analog Waveforms Recording	10-3-6
Analysis/Calculation Overview	9-2
[Analysis Report] Window	8-38
Anes	
Changing the Settings	7-3-26
Setting the Scale	7-3-26, 7-3-27
Anesthetic Gas Display and Accuracy Checks ..	7-2-15
Anesthetic Gas Monitoring	7-2-10
Changing the Settings	7-2-11
[Detailed Settings] Window	7-2-13
[Readings] Window	7-2-12
Preparation	7-2-11
Arrhythmia Event Recording	10-2-23
Arrhythmia Recall	8-29
[Recall] Window	8-30

[Zoom In] Window	8-31
Arrhythmia Recall Recording	10-2-21
Assembling the Blood Pressure Measuring Device [Press]	7-1-77
Assembling the Measuring System [CO]	7-2-36
Assigning the Functions to the Function Keys	4-7
[Averaged Waves] Window	8-37

B

Backup Battery	4-10
Basic Configuration	1-4
Basic Operation	1-6
Batteries	
Installing in the Remote Control	4-5
Battery Pack Cautions	
Charging Cautions	4-12
Operating Cautions	4-12
BIS Monitoring	7-2-3
Changing the Settings	7-2-7
[Basic Settings] Window	7-2-7
[Detailed Settings] Window	7-2-9
[Impedance Check] Window	7-2-8
Preparation	7-2-6
Blood Pressure Line Label	7-1-85
Blood Pressure Transducer	7-1-78

C

Calibration [O ₂]	7-2-20
Cautions for BIS Monitoring	7-2-3
CCO Monitoring (PiCCO Monitor)	7-3-8
Changing Alarms Settings	6-11
Changing Settings for CO Measurement	7-2-37
Changing the Arrhythmia Alarm Settings	6-14
Changing the Arrhythmia Alarm Settings Individually	6-15
Changing the Drug Settings	9-7
Changing the Height or Weight [CO]	7-2-38
Changing the Hemo Graph Settings	8-46
Changing the Interbed Alarm Settings	11-7
Changing the Settings	
Anesthetic Gas	
[Detailed Settings] Window	7-2-13
BIS	7-2-7
[Basic Settings] Window	7-2-7
[Detailed Settings] Window	7-2-9
[Impedance Check] Window	7-2-8
CO	7-2-37
Changing the Height or Weight	7-2-38
Setting the Computation Constant	7-2-37
CO ₂	7-1-59

[Basic Settings] Window	7-1-59	Clock Accuracy	4-16
[Detailed Settings] Window	7-1-61	CO	
[Gas Mixture] Window	7-1-60	Starting Waveform Monitoring	7-2-38
ECC	7-1-7	CO ₂ Monitoring	7-1-50
[Arrhy Analysis] Window	7-1-10	Adjusting Zero Balance (When Using TG-950P, TG-970P, TG-980P)	7-1-57
[Arrhyth Alarm] Window	7-1-8	Changing the Settings	
[Basic Settings] Window	7-1-8	[Basic Settings] Window	7-1-59
[Detailed Settings] Window	7-1-13	[Detailed Settings] Window	7-1-61
[ST Alarm] Window	7-1-10	[Gas Mixture] Window	7-1-60
[ST Point] Window	7-1-12	Inspection of Measuring Accuracy	7-1-61
NIBP	7-1-72	Checking Measuring Accuracy Daily	7-1-61
[Basic Settings] Window	7-1-72	Checking Measuring Accuracy More Precisely	7-1-61
[Detailed Settings] Window	7-1-74	Warnings and Cautions	7-1-57
[Program Mode] Window	7-1-73	Using TG-900P, TG-920P	7-1-58
[Venous Puncture] Window	7-1-73	Using TG-950P, TG-970P, TG-980P	7-1-58
O ₂	7-2-23	CO Measurement [CO]	7-2-35
[Basic Settings] Window	7-2-23	Assembling the Measuring System	7-2-36
Press	7-1-81	Changing Settings for CO Measurement	7-2-37
[Basic Settings] Window	7-1-82	Changing the Height or Weight	7-2-38
[Cursor] Window	7-1-83	Setting the Computation Constant	7-2-37
[Detailed Settings] Window	7-1-83	Entering Blood Pressure Values	7-2-39
[Labels] Window	7-1-82	Inserting and Retaining the Catheter	7-2-38
[Scale] Window	7-1-82	Measuring the Pulmonary Capillary Wedge	
[Temporary Labels] Window	7-1-82	Pressure	7-2-40
RESP	7-1-48	Registering in the Hemo List	7-2-43
[Basic Settings] Window	7-1-48	Starting CO Measurement	7-2-41
[Detailed Settings] Window	7-1-49	Starting Waveform Monitoring	7-2-38
SpO ₂	7-1-21, 7-1-31, 7-1-41	CO Measurement Results Recording	10-3-4
[Basic Settings] Window	7-1-21, 7-1-32, 7-1-41	Computation Constant [CO]	7-2-37
[Detailed Settings] Window	7-1-22, 7-1-32, 7-1-42	Connecting to the Respiration Circuit [O ₂]	7-2-22
Temp	7-1-90	CU-191R, CU-192R Core Unit	
[Basic Settings] Window	7-1-91	Cleaning the Surface	14-11
[Detailed Settings] Window	7-1-91	Disinfecting the Surface	14-12
[Labels] Window	7-1-91	Cursor	
TOF	7-2-50	Measuring with the Cursor	7-1-83
AF-101P	7-2-50	Setting the Sweep Speed	7-1-83
Neuromuscular Transmission Monitor	7-2-53		
Changing the Upper/Lower Limit Alarm Settings ..	6-12	D	
Setting the Upper/Lower Limits Automatically	6-13	DBS Mbde	7-2-48
Setting the Upper/Lower Limits Individually	6-13	Deactivating Alarms	6-7
Channel Settings	4-6	Default Settings	15-2
Checking the Dominant QRS	7-1-5	Determination of Applied Parts	2-19
Cleaning		Disabling the Touch Keys	7-0-13
Air Hoses	14-16	Discharging a Patient	5-4
Electrode Leads and ECG Connection Cords	14-15	Disinfecting	
QE-910P BIS Processor/BISx	14-25	Air Hoses	14-16
Connection Cables	14-25	Electrode Leads, ECG Connection Cords and	
RY-910PA Remote Controller (Optional)	14-15	SpO ₂ Connection Cords	14-15
SpO ₂ Connection Cords	14-15	QE-910P BIS processor/BISx	14-25
Cleaning and Disinfection		Connection Cables	14-25
AE-918P Neuro Unit	14-25		
AE-920P Neuro Unit	14-25		
WS-960P Recorder Unit	14-13		
Cleaning the Sensor (Recorder Unit)	14-14		
Cleaning the Thermal Head (Recorder Unit)	14-13		

RY-910PA Remote Controller (Optional).....	14-15
Displaying the Menu Window.....	1-8
Displaying the Short Trend.....	7-0-6
Disposal	
AG-400RK CO ₂ Unit Filterline.....	14-9
BIS Sensor.....	14-10
Exhaust Gas Tube of AG-920RK Multigas Unit....	14-10
Flow Tube of Multigas/Flow Unit.....	14-10
Sampling Line, T-piece and Water Trap of Multigas Unit.....	14-10
Double Burst Stimulation.....	7-2-48
Drug Calculation.....	9-6
Changing the Settings.....	9-7
Registering a Drug.....	9-8
Selecting a Drug.....	9-6
Viewing the Titration Table.....	9-7

E

ECG Analysis.....	9-3
Starting Analysis.....	9-5
ECG Analysis Results Recording.....	10-2-24
ECG Monitoring [ECG].....	7-1-3
Changing the Settings.....	7-1-7
[Arrhy Analysis] Window.....	7-1-10
[Arrhyth Alarm] Window.....	7-1-8
[Basic Settings] Window.....	7-1-8
[Detailed Settings] Window.....	7-1-13
[ST Alarm] Window.....	7-1-10
[ST Point] Window.....	7-1-12
Checking the Dominant QRS.....	7-1-5
Learning the ECG.....	7-1-5
Preparation.....	7-1-3
Warnings and Cautions.....	7-1-4
ECG Recording.....	10-1-6
EEG Trend Recording.....	10-2-34
Entering Blood Pressure Values [CO].....	7-2-39
Exhaust Gas Adapter	
Cleaning and Disinfection.....	14-16
Exiting Sleep Mode.....	7-0-13

F

Factory Default Settings.....	15-2
Flow of Alarm Function.....	6-6
Flow/Paw and Loop Recording.....	10-3-4
Freezing Waveforms.....	7-0-9
Full Disclosure.....	8-18
[Compressed Traces] Window.....	8-19
[Zoom In] Window.....	8-20
Full Disclosure Recording.....	10-2-15

G

General.....	1-2
--------------	-----

H

Hemodynamics Graph.....	8-42
[Target Graph] Window.....	8-45
[Trend + Target] Window.....	8-43
[Trend] Window.....	8-44
Hemodynamics List Recording.....	10-2-10
HiQ-View Function.....	12-17
Home Screen.....	7-0-4
Displaying the Short Trend.....	7-0-6
Freezing Waveforms.....	7-0-9
Unfreezing Waveforms.....	7-0-9
Using Sleep Mode.....	7-0-12

I

IBP Monitoring [Press]	
Assembling the Blood Pressure Measuring Device.....	7-1-77
Changing the Settings.....	7-1-81
[Basic Settings] Window.....	7-1-82
[Detailed Settings] Window.....	7-1-83
[Labels] Window.....	7-1-82
[Scale] Window.....	7-1-82
[Temporary Labels] Window.....	7-1-82
Selecting a Blood Pressure Measuring Device.....	7-1-76
Setting the Blood Pressure Transducer.....	7-1-78
Impedance Method [RESP].....	7-1-46
Indications for Use.....	1-3
Individual Bed Window [Interbed].....	11-3
Inserting and Removing the Input Unit.....	4-3
Inserting and Retaining the Catheter [CO].....	7-2-38
Inserting the Input Unit into the Data Acquisition Unit.....	4-3
Interbed Alarm.....	6-17
Interbed Alarm Function.....	11-6
Interbed Overview.....	11-2
Interface	
Cleaning and Disinfection.....	14-17

J

JA-920P Interface Unit.....	2-16
-----------------------------	------

L

Learning the ECG.....	7-1-5
Loading the Recording Paper.....	4-2

Lung Function Calculation Recording.....	10-3-10
Lung Function List.....	8-15
Lung Function Recording	10-2-12

M

Manual Recording	10-1-2
Measurement and Setting Items	
Anes.....	7-3-22
Measurement [NIBP].....	7-1-68
Interval Measurement.....	7-1-68
Manual Measurement.....	7-1-68
SIM Measurement.....	7-1-68
STAT Measurement.....	7-1-68
Measuring the PCWP [CO].....	7-2-40
Measuring the Pulmonary Capillary Wedge Pressure [CO].....	7-2-40
Measuring Two SpO ₂ Values ..	7-1-18, 7-1-29, 7-1-37
Menu Window Operations	1-8
Monitoring Overview.....	7-0-2
Monitoring Preparation	
Anesthetic Gas.....	7-2-11
BIS.....	7-2-6
ECG.....	7-1-3
O ₂	7-2-20
RESP.....	7-1-46
SpO ₂	7-1-18, 7-1-28, 7-1-37
When Using AY-631P, AY-633P, BSM-1733	7-1-37, 7-1-38
When Using AY-651P, AY-653P, BSM-1753	7-1-28
Temp.....	7-1-88
TOF.....	7-2-49
Multigas Unit	
Cleaning the Exhaust Gas Adapter.....	14-18
Connection Cables.....	14-18
Disinfecting the Water Trap.....	14-19
Draining the Water Trap.....	14-19, 14-20

N

NIBP List.....	8-13
NIBP List Recording	10-2-8
NIBP Measurement [NIBP].....	7-1-62
Changing the Settings.....	7-1-72
[Basic Settings] Window	7-1-72
[Detailed Settings] Window	7-1-74
[Program Mode] Window	7-1-73
[Venous Puncture] Window.....	7-1-73
Measurement	7-1-68
Interval Measurement.....	7-1-68
Manual Measurement	7-1-68
PWTT Triggered Measurement.....	7-1-70
SIM Measurement.....	7-1-68

STAT Measurement.....	7-1-68
Warnings and Cautions.....	7-1-67

O

O ₂ Monitoring.....	7-2-20
Calibration.....	7-2-20
Changing the Settings.....	7-2-23
[Basic Settings] Window	7-2-23
Connecting to the Respiration Circuit.....	7-2-22
Preparation.....	7-2-20
OCRG	8-64
OCRG Recording	10-3-2
OCRG Review Recording.....	10-2-36
Operating Methods	1-6
Operating with a Mouse	1-6
Operating with a Remote Control	1-6
Using the Touch Screen and Operating Keys.....	1-6
Operating with a Mouse.....	1-6
Operating with a Remote Control	1-6
Operations at Alarm Occurrence	6-4
Out of Paper	10-0-6

P

Panel Description	
AA-672P, AA-674P Smart Expansion Unit.....	2-10
AY-631P, AY-633P, AY-651P, AY-653P, AY-660P, AY-661P, AY-663P, AY-671P, AY-673P Input Unit	2-6
JA-690PA, JA-694PA Data Acquisition Unit	2-12
RY-910PA Remote Controller	2-18
Paper Magazine is Open.....	10-0-6
Periodic Recording Interval	10-0-8
Periodic Replacement Parts	
Multigas Unit.....	14-9
Periodic Replacement Parts (Standard Configuration).....	14-5, 14-8
PK Simulation Graphs	8-48, 8-54
[Graph] Window.....	8-48
[List] Window.....	8-52
[PK Sim Settings] Window	8-49
References for PK Simulation Graphs	
BIS Color Trend Component	8-55
Drugs and Models.....	8-55
Equation for Blood Concentration (C1) and Effect Site Concentration (Ce).....	8-54
Parameter Equations.....	8-57
PK Simulation Graph Settings	8-53
Post-Tetanic Count Stimulation	7-2-46
[Printer] Window.....	10-0-8
PTC Mbde	7-2-46

R

Recorded Data	10-0-5
Recorder Unit	
Recorded Data	10-0-5
[Recorder] Window.....	10-0-7
Recording	
12-lead Analysis Results	10-2-24
12-Lead Waveforms	10-3-7
Alarm.....	10-1-4
Alarm History.....	10-2-19
Analog Waveforms.....	10-3-6
Arrhythmia Event Graphs	10-2-23
Arrhythmia Recall Waveforms.....	10-2-21
Bedside Monitor Operation.....	10-0-6
CO Measurement Results.....	10-3-4
Drug Calculation	10-3-9
EEG Trend Data	10-2-34
Flow/Paw and Loop.....	10-3-4
Full Disclosure Waveforms	10-2-15
Hemodynamics Graph.....	8-47
Hemodynamics List.....	10-2-10
Histogram.....	10-2-3
Lung Function Calculation.....	10-3-10
Lung Function List.....	10-2-12
Manual.....	10-1-2
NIBP List.....	10-2-8
OCRG.....	10-3-2
OCRG Review Data.....	10-2-36
Recorded Data	10-0-5
ST Recall Waveforms	10-2-17
ST Review Data.....	10-2-18
TOF List.....	10-2-14
TOF Measurement Results.....	10-3-5
Trend Graphs.....	10-2-2, 10-2-32
Vital Sign List.....	10-2-6
Recording Output Destination	10-0-2
Recording Settings	10-0-7
Recording Speed.....	10-0-7, 10-0-8
Recording Types.....	10-0-2
Recording Waveforms	10-0-7, 10-0-8
Registering a Drug.....	9-8
Registering in the Hemo List [CO].....	7-2-43
Remote Control Channel Label	
Attaching the Remote Control Channel Label to the Display of the Monitor	4-6
Remote Controller	
Assigning the Functions	4-7
Attaching the Remote Control Channel Label to the Display of the Monitor	4-6
Installing the Batteries.....	4-5

Preparation.....	4-5
Setting the Channel	4-6
Respiration Calculation.....	9-9
Entering Values in the Entry Items.....	9-9
Viewing the Calculation Results	9-9
Respiration Monitoring [RESP].....	7-1-46
Changing the Settings.....	7-1-48
[Basic Settings] Window	7-1-48
[Detailed Settings] Window	7-1-49
Impedance Method.....	7-1-46
Preparation.....	7-1-46
Thermistor Method.....	7-1-47
Restoring Touch Key Operation.....	7-0-14
Review Overview.....	8-3
rSO ₂ /StO ₂ Monitoring.....	7-2-54

S

Scale	
Anes.....	7-3-26, 7-3-27
Vent.....	7-3-38
Screen Messages.....	13-3
Selecting a Blood Pressure Measuring Device	
[Press]	7-1-76
Service.....	14-26
Short Trend Scale	
MV.....	7-3-28, 7-3-40
RR.....	7-3-28, 7-3-40
Showing the Admit Confirmation Window.....	4-14
Signal IQ (When Using AY-631P, AY-633P Only)	
[SpO ₂].....	7-1-40
Socket Pin Assignment.....	17-78
SpO ₂ Label	7-1-24, 7-1-33, 7-1-44
SpO ₂ Monitoring	
Changing the Settings.....	7-1-21, 7-1-31, 7-1-41
[Basic Settings] Window	7-1-21, 7-1-32, 7-1-41
[Detailed Settings] Window	7-1-22, 7-1-32, 7-1-42
[Labels] Window.....	7-1-24, 7-1-33, 7-1-44
Measuring Two SpO ₂ Values	7-1-18, 7-1-29, 7-1-37
Preparation.....	7-1-18, 7-1-28, 7-1-37
When Using AY-631P, AY-633P, BSM-1733	7-1-37, 7-1-38
When Using AY-651P, AY-653P, BSM-1753	7-1-28
Warnings and Cautions.....	7-1-19, 7-1-30, 7-1-38
Starting CO Measurement.....	7-2-41
Starting Waveform Monitoring [CO].....	7-2-38
Storing Recording Paper	14-14
ST Recall.....	8-23
ST Recall Recording.....	10-2-17
ST Review Recording.....	10-2-18
Symbols.....	1-10

T

tcPO ₂ /tcPCO ₂	
Changing the Settings.....	7-3-42
Temperature Measurement Site Label	7-1-92
Temperature Monitoring [Temp].....	7-1-88
Changing the Settings.....	7-1-90
[Basic Settings] Window	7-1-91
[Detailed Settings] Window	7-1-91
[Labels] Window.....	7-1-91
[Temporary Labels] Window	7-1-91
Preparation.....	7-1-88
Temp Measurement.....	7-2-51
Temporary Blood Pressure Line Label	7-1-87
Temporary Temperature Measurement Site Label.....	7-1-93
Tetanic Stimulation	7-2-47
TET Mode.....	7-2-47
Thermistor Method [RESP].....	7-1-47
Timing of Gas Calibration for Anesthetic Gas....	7-2-15
TOF Measurement Results Recording.....	10-3-5
TOF Mode	7-2-46
TOF Monitoring	
Changing the Settings	
Using AF-101P	7-2-50
Checking the TOF Window.....	7-2-52
Using AF-101P	7-2-52
Preparation	
AF-101P.....	7-2-49
Neuromuscular Transmission Monitor.....	7-2-49
Stimulation Modes	7-2-46
Terms.....	7-2-48
Train-Of-Four Consecutive Stimulations	7-2-46
Transport Function	12-2
Patient Authentication	12-8
Preparing for Input Unit Removal.....	12-10
Saved Data.....	12-5
Sending Data to the Central Monitor	12-7
Trend Graph	8-6
[Histogram] Window.....	8-7
Trend Graphs Recording.....	10-2-2 10-2-32
Trend Graph Types and Scale.....	8-7
Trend List	
Lung List.....	8-15
NIBP List.....	8-13
TOF List	8-16
Turning the Monitor Off.....	4-15
Turning the Monitor On.....	4-13

U

Unfreezing Waveforms.....	7-0-9
Using Sleep Mode	7-0-12
Using the Car Seat Challenge Function	7-0-14
Using the Touch Screen and Operating Keys.....	1-6

V

Vent	
Changing the Settings.....	7-3-38
Checking the Settings.....	7-3-39
Setting the Scale.....	7-3-38
Viewing the Interbed Bed Window	
Individual Bed Window.....	11-3
View Other Beds Window	11-2
Viewing the Titration Table.....	9-7
View Other Beds Window [Interbed].....	11-2
Vital Sign List.....	8-12
List.....	8-12
Vital Sign List Types	8-13
Vital Sign List Recording	10-2-6

W

Warnings and Cautions	
CO ₂	7-1-57
Using TG-900P, TG-920P.....	7-1-58
Using TG-950P, TG-970P, TG-980P	7-1-58
ECG	7-1-4
NIBP.....	7-1-67
SpO ₂	7-1-19, 7-1-30, 7-1-38
TOF.....	7-2-44
Using AF-101P	7-2-44
WS-960P Recorder Unit	
Cleaning and Disinfection.....	14-13
Disinfecting and Cleaning the Panel	14-13
Loading the Recording Paper	4-2
Panel Description	2-17
Storing Recording Paper	14-14

Y

YL-920P Alarm Indicator.....	2-5
------------------------------	-----



Manufacturer

NIHON KOHDEN CORPORATION

1-31-4 Nishiochiai, Shinjuku-ku Tokyo 161-8560, Japan
Phone +81 3-5996-8041

NIHON KOHDEN ITALIA S.r.l.

Via Fratelli Bronzetti 28, 24124 Bergamo, Italy
Phone +39 035-219543
Fax +39 035-232546

North and South America

NIHON KOHDEN AMERICA, INC.

15353 Barranca Parkway, Irvine, CA 92618, U.S.A.
Toll-free +1-800-325-0283
Phone +1 949-580-1555
Fax +1 949-580-1550

NIHON KOHDEN UK LTD.

Trident Court 118, 1 Oakcroft Road
Chessington, Surrey KT9 1BD, UK
Phone +44 20-8391-6800
Fax +44 20-8391-6809

NIHON KOHDEN MEXICO S.A. DE C.V.

Insurgentes Sur 730 Piso 9 Oriente, Col. Del Valle
C.P. 03100 Delegacion Benito Juarez, Ciudad de Mexico
Phone +52 55-8851-5550
Fax +52 55-8851-5580

Asia

SHANGHAI KOHDEN

MEDICAL ELECTRONIC INSTRUMENT CORP.

No. 567 Huancheng Bei Road
Shanghai Comprehensive Industrial Development Zone
Fengxian District, Shanghai 201401, China
Phone +86 21-5743-6998
Fax +86 21-5743-6939

NIHON KOHDEN DO BRASIL LTDA.

Rua Diadema, 89, 1º andar, conjuntos 11 a 17, bairro Mauá
no Município de São Caetano do Sul, Estado de São Paulo
CEP 09580-670 Brasil
Phone +55 11-3044-1700
Fax +55 11-3044-0463

NIHON KOHDEN SINGAPORE PTE LTD

1 Maritime Square, #10-34 HarbourFront Centre
Singapore 099253
Phone +65 6376-2210
Fax +65 6376-2264

Europe

EC REP European Representative

NIHON KOHDEN EUROPE GmbH

Raiffeisenstrasse 1Q D-61191 Rosbach, Germany
Phone +49 6003-827-0
Fax +49 6003-827-599

NIHON KOHDEN INDIA PVT. LTD.

308 Tower A, Spazedge, Sector 47, Sohna Road
Gurgaon-122002 Haryana, India
Toll-free +91 1800-103-8182
Phone +91 124-493-1000
Fax +91 124-493-1029

NIHON KOHDEN DEUTSCHLAND GmbH

Raiffeisenstrasse 1Q D-61191 Rosbach, Germany
Phone +49 6003-827-0
Fax +49 6003-827-599

NIHON KOHDEN MIDDLE EAST FZE

JAFZA One Tower A, 19th floor, Office No. 1912
P.O. Box 261516 Jebel Ali Free Zone, Dubai, U.A.E.
Phone +971 4-884-0080
Fax +971 4-880-0122

NIHON KOHDEN FRANCE SARL

8 rue Francois Delage, 94230 Cachan, France
Phone +33 1-49-08-05-50
Fax +33 1-49-08-93-32

NIHON KOHDEN KOREA, INC.

3F, Cheongok Bldg., 88 Dongmak-ro, Mapo-gu,
Seoul, 04075 Republic of Korea
Phone +82 2-3273-2310
Fax +82 2-3273-2352

NIHON KOHDEN IBERICA S.L.

C/ Ulises 75A, E-28043 Madrid, Spain
Phone +34 917-161-080
Fax +34 913-004-676

Contact information is accurate as of February 2019. Visit <https://www.nihonkohden.com/> for the latest information.

The model and serial number of your device are identified on the rear or bottom of the unit.

Write the model and serial number in the spaces provided below. Whenever you call your representative concerning this device, mention these two pieces of information for quick and accurate service.

Model _____

Serial Number _____

Your Representative

Note for users in the territory of the EEA and Switzerland:

Any serious incident that has occurred in relation to the device should be reported to the European Representative designated by the manufacturer and the Competent Authority of the Member State of the EEA and Switzerland in which the user and/or patient is established.

CSM-1901_0614-906759W

MUMG 249 [A] - CO-2564



NIHON KOHDEN CORPORATION

1-31-4 Nishiochiai, Shinjuku-ku, Tokyo 161-8560, Japan
Phone +81 3-5996-8041
<https://www.nihonkohden.com/>