


NovaGuard® SA Pro Needle Safety System Instructions For Use

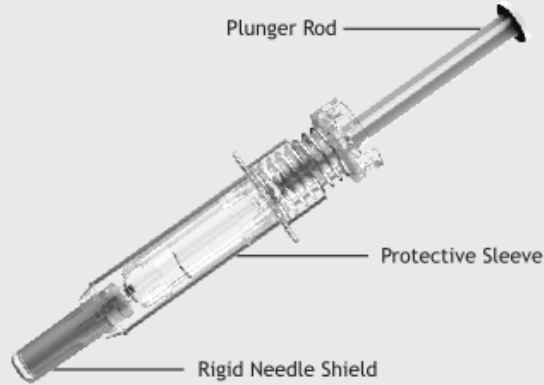

Available in 1.0mL long and 0.5mL device variants.

Before You Begin

Single-use device indicated for use as an accessory with:

- Pre-filled ISO 11040-4 Standard 0.5mL glass syringes with a maximum needle length of 1/2" (12.7mm), or
- Pre-filled ISO 11040-4 Standard 1mL long glass syringes with a maximum needle length of 5/8" (15.9mm)

to aid in the protection of healthcare professionals, caregivers and patients who self-inject doctor prescribed medications and individuals that assist self-injecting patients, from accidental needlestick injuries. Intended for use in clinical or home care settings.

Preparation

Wash Hands




Choose Recommended
Injection Site



Swab Site




1. Remove Rigid Needle Shield

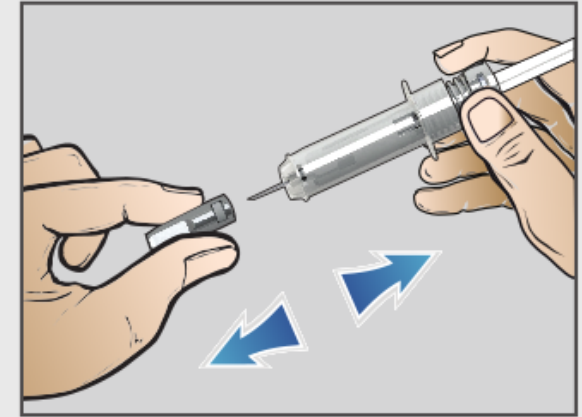


Remove rigid needle shield from the device.

2. Injection

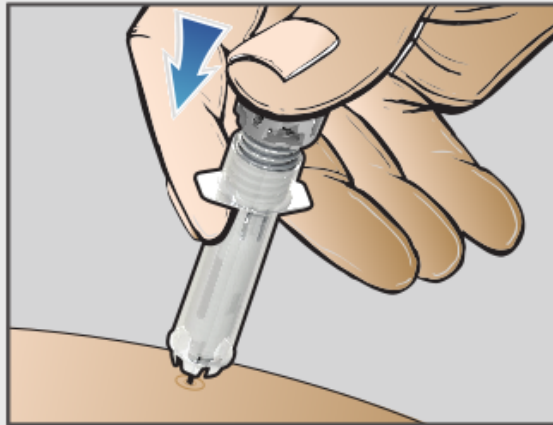


Insert needle into injection site and give injection as normal.

3. Safety Activation

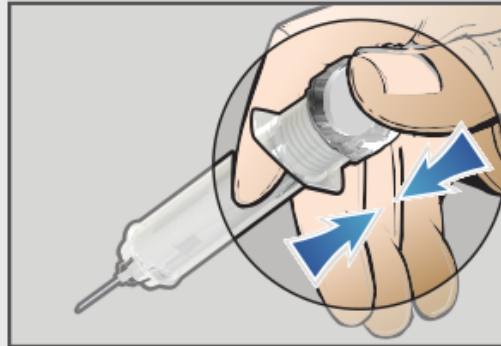

Remove from injection site and depress plunger rod until the device is fully compressed, then remove thumb to activate the protective sleeve. The protective sleeve will automatically cover the needle and an audible "click" will be heard to confirm protective sleeve activation.

4. Disposal

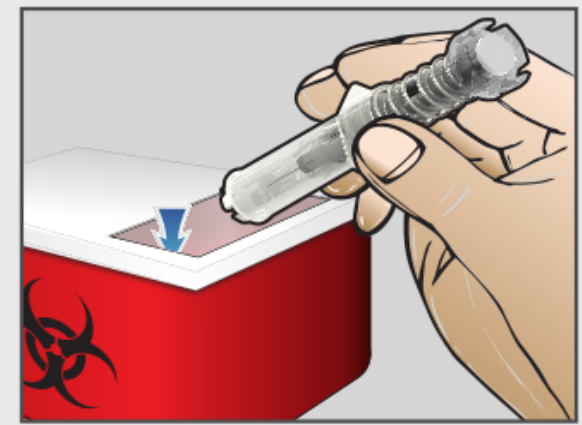

Once protective sleeve is activated, dispose of unit in an approved sharps container.

▲ Precautions

Do not use if package is damaged or if the product has been dropped. Do not replace the rigid needle shield after removing. Do not reuse device. Dispose of used device in accordance with applicable regulations.

▲ Cautions

Federal (USA) law restricts this device to sale by or on the order of a physician. This device is intended to be used for one patient only.

West
By your side
for a healthier world™

West and the diamond logo, By your side for a healthier world™ and NovaGuard® are registered trademarks or trademarks of West Pharmaceutical Services, Inc., in the United States and other jurisdictions.

Manufactured for: Tech Group Europe Limited
Damastown Industrial Park
Dublin 15, IRELAND

Date: Nov-2021
DHF-110034 Rev 07