


<https://www.ulabsystems.com/the-ulab-advantage/>



## How It Works

Open your patient's intra-oral scan directly in uLab software. Program the desired treatment plan, which can include clear aligners in as little as 10 minutes. Design your patient's aligners and fabricate them with many of the 3D printers on the market. Our automated uContour machine will process (cut and initial polish) your 3D model with precision in about a minute. Create a seamless patient experience by custom-branding your aligners with the name of your practice.

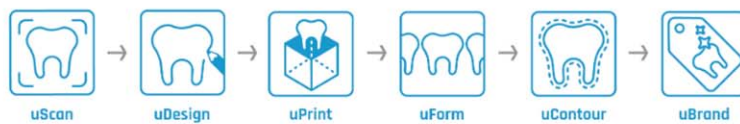

HOW WE WORK WITH YOU




<https://www.ulabsystems.com/how-we-work-with-you/>



Now let's watch  
an end to end case  
that takes less than 5 minutes

▶ 1:33 / 8:00




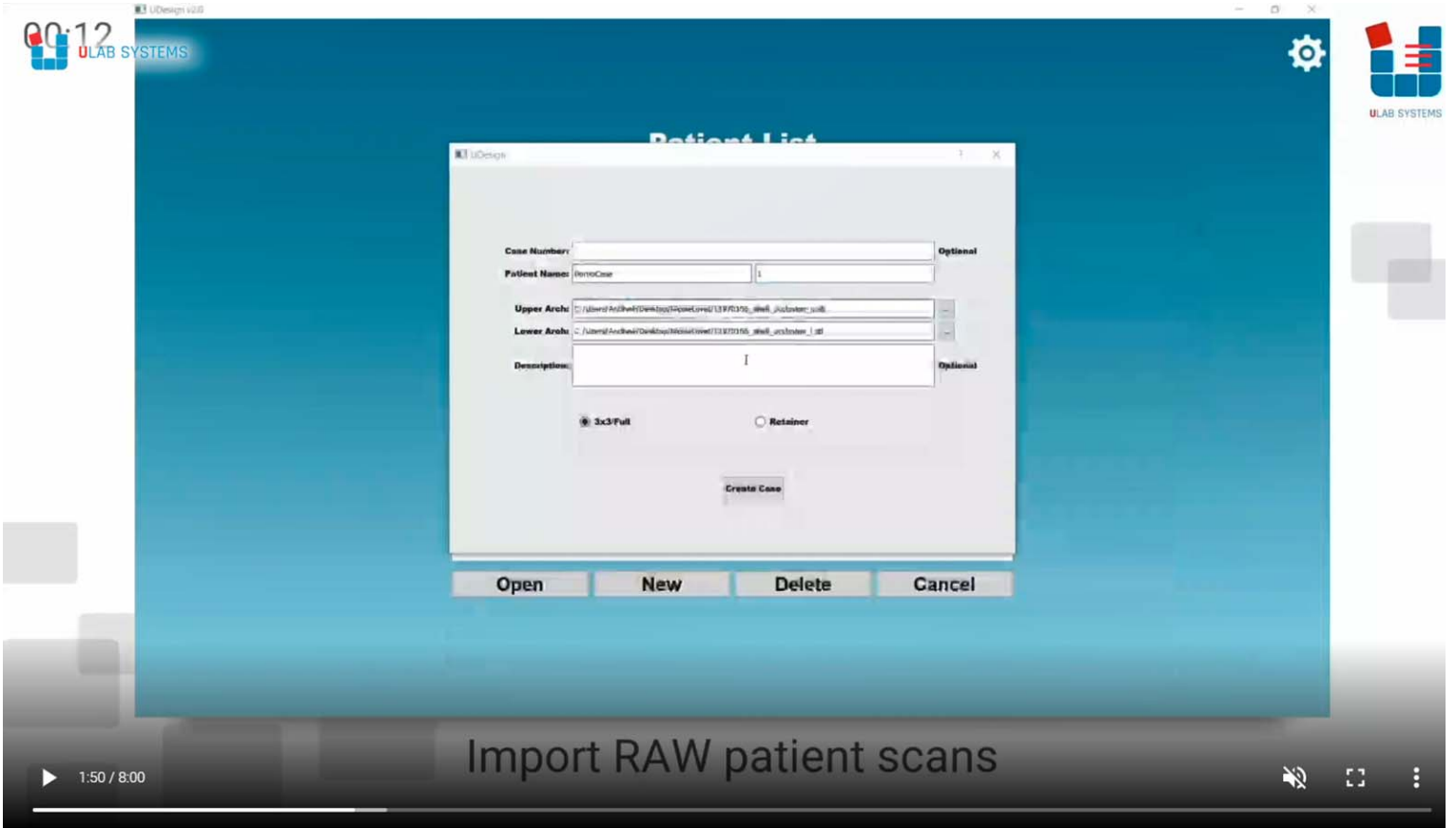

This screenshot shows the 'Patient List' dialog box in the UDesign software. The dialog box contains the following fields and options:

- Case Number:** An empty text field with an 'Optional' label to its right.
- Patient Name:** A text field containing 'DemoCase' and a numeric field containing '1'.
- Upper Arch:** A dropdown menu showing a file path: 'C:\Users\Administrator\Desktop\3D\3D\120\_001\_01\01\_upper\_001\_01\_001.stl'.
- Lower Arch:** A dropdown menu showing a file path: 'C:\Users\Administrator\Desktop\3D\3D\120\_001\_01\01\_lower\_001\_01\_001.stl'.
- Description:** A text field containing 'I' with an 'Optional' label to its right.
- Options:** Radio buttons for '3x3 Full' (selected) and 'Retainer'.
- Buttons:** 'Create Case' (bottom center), 'Open', 'New', 'Delete', and 'Cancel' (bottom row).


The video player interface at the bottom shows a play button, a progress indicator at 1:50 / 8:00, and the title 'Import RAW patient scans'.

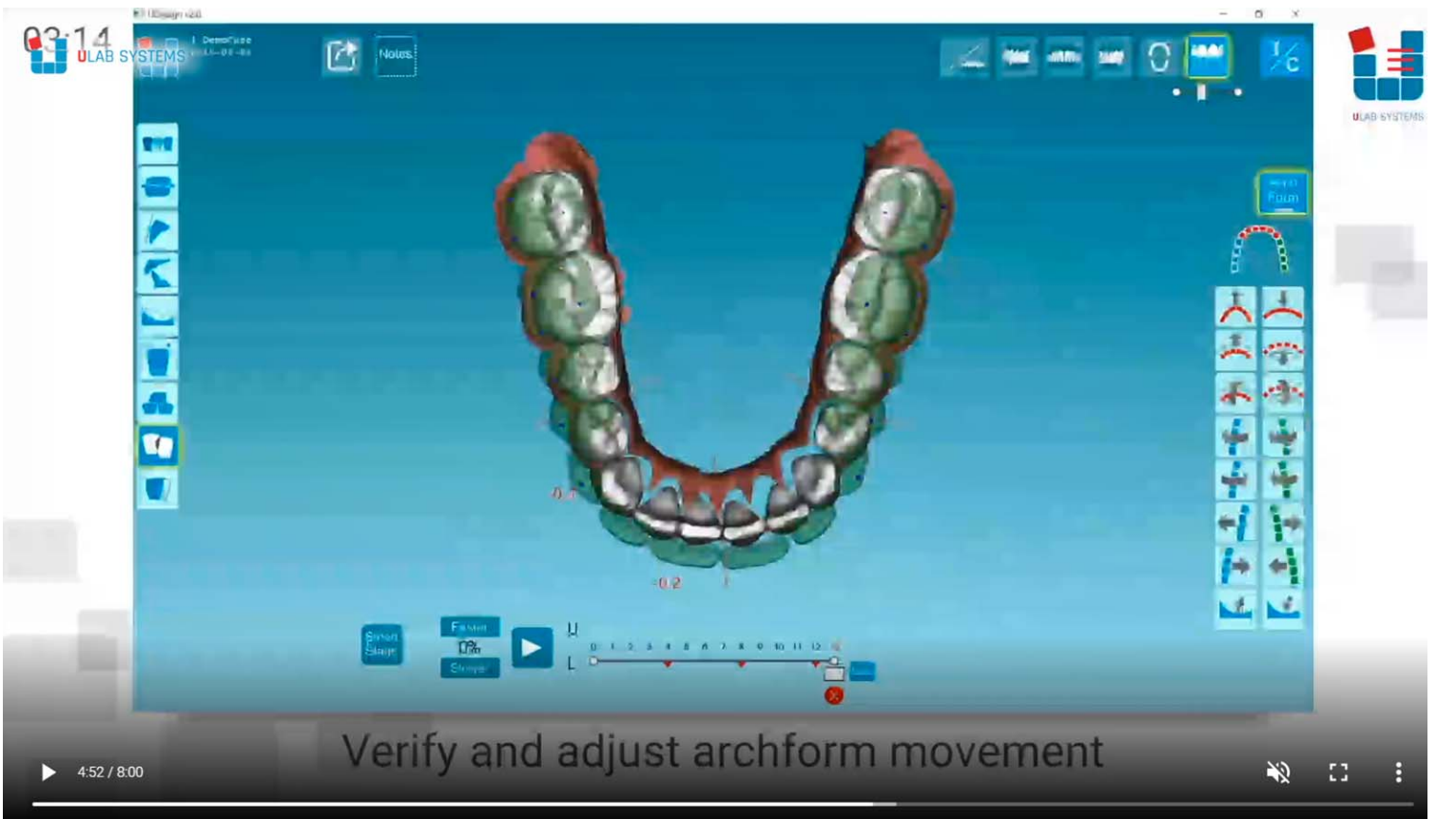

This screenshot shows the main 3D modeling interface of the UDesign software. The central focus is a 3D model of a dental arch, colored in green and red. The interface includes:

- Top Bar:** Contains various icons for file operations and settings.
- Left Panel:** A vertical toolbar with icons for different views and functions.
- Right Panel:** A grid of icons for arch movement and alignment, including a 'MESH FLUID' button.
- Bottom Panel:** A 'Simulation' section with 'Restart', 'Pause', and 'Stop' buttons, along with a progress slider set to 0%.

The video player interface at the bottom shows a play button, a progress indicator at 4:52 / 8:00, and the title 'Verify and adjust archform movement'.



Product is also available in

**The uLab Advantage**    **Patient: Get Started**

Leverage the power of the most  
**How We Work With**    **Orthodontist: Get**  
**You**    **Started**

Streamline and make your  
**Our Expertise**    **uNews**  
 your hands become hands  
 where we believe it belongs. No  
 third-party production  
 companies, long wait times or  
 excessive lab fees. There's  
 custom treatment plans that  
 offer the latest advances of  
 same day aligners and the best  
 possible outcomes for your  
 patients.

**CONTACT**  
 uLab Systems Inc.  
 3 Lagoon Drive, Suite 180  
 Redwood City, CA 94065  
 866-900-8522  
[info@ulabsystems.com](mailto:info@ulabsystems.com)

**CAREERS**  
 For a list of current open positions,  
 please check out our [careers](#) page!

**FLEXIBLE TREATMENT PLANS**

**STATE-OF-THE-ART TECHNOLOGY**



## Find a uLab-Trusted Orthodontist Near You

Orthodontists who use uLabs want to bring the best service and most effective care to their patients. See if your orthodontist uses uLab Systems or find a trained practitioner near you to get started.

**YOUR FIRST NAME \***

**YOUR ZIP CODE \***

**YOUR LAST NAME \***

**YOUR EMAIL \***

**YOUR ORTHODONTIST'S NAME**

**YOUR ORTHODONTIST'S PHONE**

**YOUR ORTHODONTIST'S WEBSITE**

**GET STARTED**

**CONTACT**

uLab Systems Inc.  
3 Lagoon Drive, Suite 180  
Redwood City, CA 94065  
866-900-8522  
[info@ulabsystems.com](mailto:info@ulabsystems.com)

**CAREERS**

For a list of current open positions, please check out our [careers](#) page!

**uNEWS**

Stay up to date with the latest through our [uLab news](#) page!