WARNING: These instructions contain important information for safe use of the product. Read the entire contents of these instructions for use, including warnings and cautions, before using this product. Failure to properly follow warnings, cautions and instructions could result in death or serious injury to the patient.

Description:

The Snuggly Warm® Adult Full Body Convective Warming Blanket is a disposable, single-use, latex-free blanket that is placed over the patient, and is connected to a convective warmer. The convective warmer generates heated air that is distributed throughout the blanket to warm the patient.

Indications for Use

The convective warming blanket is intended for use with a convective warmer for thermal regulation of a patient's temperature and is designed to prevent hypothermia and/or reduce cold discomfort during and after surgical procedures. This product is intended for use by appropriately trained healthcare professionals in clinical environments.

CONTRAINDICATIONS

- Thermal injury may occur if convective warming therapy is applied to lower extremities during aortic cross-clamping procedures.
- Thermal injury may occur if convective warming therapy is applied to ischemic limbs.

WARNINGS

Death or serious injury may occur to the patient or users if these warnings are not followed:

- This convective warming blanket can only be used in conjunction with the EQUATOR® convective warmer (EQ-5000) and the Snuggly Warm™ convective warmer (SW-4000). For the safe operation of this blanket, the user must follow all warnings, cautions, and instructions provided in the Instructions for Use supplied with the compatible convective warmer. In addition to these Instructions for Use.
- Always start therapy on the lowest non-ambient temperature setting to prevent thermal injury. Increase the temperature setting, if required, using core body temperature and cutaneous response of skin in contact with the convective warming blanket as indicators.
- To prevent thermal injury, do not use the highest temperature setting when treating patients who have decreased sensation, are nonsensate, or have poor perfusion.
- Monitor patient's body core temperature, vital signs, and observe cutaneous response at regular intervals to prevent thermal injury. If erythema or instability in vital signs is evident, decrease the temperature setting or discontinue use of convective warming therapy.
- To prevent thermal injury in hypotensive or hypoperfused patients, observe cutaneous response more frequently. Reduce the temperature setting or discontinue use of convective warming therapy if instability in vital signs or erythema occurs.
- If the patient's nose or mouth is positioned against the convective warming blanket or the plastic drape, closely monitor the patient's breathing to prevent suffocation injury.
- Cover all open wounds in contact with the convective warming blanket to prevent airborne contamination.
- The hose nozzle MUST be connected to a Snuggly Warm™ convective warming blanket. Do not treat patients with the hose alone. Thermal injury may occur.
- Use only one convective warmer and one hose. Use of multiple convective warmers in multiple inlet ports of the same convective warming blanket may cause damage to the blanket and thermal injury to the patient.
- To prevent thermal injury, do not allow any of the patient's body parts to rest on the active hose inlet.
- Do not place objects onto the convective warming blanket that will obstruct air flow. Items on the blanket can produce localized pressure on the patient's skin, reducing cutaneous blood flow and causing thermal injury to the patient.
- Always place the perforated side of the convective warming blanket, the side with small holes, towards the patient.
- Failure to do so may result in thermal injury.
- Do not use a convective warming blanket over transdermal medications as this may lead to increased drug delivery that may result in patient injury or death.
- This convective warming blanket is for single-use only. Reusing the blanket may increase the risk of cross contamination.
- Use with the EQUATOR™ convective warmer (EQ-5000) and the Snuggly Warm™ convective warmer (SW-4000). Using non-compatible convective warmers may cause damage to the convective warming blanket and thermal injury to the patient.
- Do not sterilize convective warming blankets. Sterilizing blankets may damage the blanket and may cause thermal injury to the patient.

CAUTIONS

Malfunction, failure, or damage to the device may occur if these cautions are not followed:

- Federal (U.S.A.) law restricts this device to sale by or on the order of a physician.
- Medical devices require specific material characteristics to perform as intended. These characteristics have been verified for single use only. Any attempt to re-process the device for subsequent re-use may adversely affect the integrity of the device or lead to deterioration in performance.

Disposal Information

Dispose of the convective warming blankets in a safe manner according to local guidelines for disposal of contaminated medical waste.

Product meets Consumer Products Safety Commission's flammable fabric regulation, 16 CFR 1616; however, follow standard safety protocols when using high-intensity heat sources.

The Smiths Medical and Level 1 design marks, and EQUATOR and Snuggly Warm are trademarks of the Smiths Medical family of companies. The symbol ® indicates the trademark is registered in the U.S. Patent and Trademark office and certain other countries. All other names and marks mentioned are the trade names, trademarks or service marks of their respective owners.

©2010 Smiths Medical family of companies. All rights reserved.

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