



BUC Tips Ultrasonics

Endodontic Ultrasonic Tips From Dr. Buchanan

BUC-1 and BUC-1A

The BUC-1 and BUC-1A can be used for gross dentin removal, moving access line angles, cutting a groove in the mesial access wall to drop in to MB2 canals, and for unroofing pulp chambers. Helpful in finding reclusive MB2s.

BUC-2 and BUC-2A

The BUC-2, with its disk-like radiused tip, can be used to plane attached pulp stones from the pulp chamber floor. In molars, it can be used to horizontally smooth the pulp chamber floor to get to the darker-colored dentin. Can be used for corners of molar access prep and bicuspoid access prep.

BUC-3 and BUC-3A

The BUC-3 and BUC-3A are extremely active instruments with sharp tips. They are used for chasing canals halfway up a root or for digging around a post or carrier-based obturator to remove it. The water port is placed near the cutting surface of the tip. Use only at lowest power level.

[Buy Now](#) [Documentation](#)

BUC Tips

975-0401 Ultrasonic Tip BUC 1	Add to Cart More Info
975-0402 Ultrasonic Tip BUC 2	Add to Cart More Info
975-0403 Ultrasonic Tip BUC 2A	Add to Cart More Info
975-0404 Ultrasonic Tip BUC 3	Add to Cart More Info
975-0406 Ultrasonic Tip BUC 1A	Add to Cart More Info
975-0407 Ultrasonic Tip BUC 3A	Add to Cart More Info

*Please note: We ONLY sell our products to licensed dental or medical practices.

*During the checkout process you will have the opportunity to select your preferred dealer. Once you submit your order, it will be sent directly to your preferred dealer and will be subject to their pricing. Thank you.

<p>Contact</p> <p>Terms of Use</p> <p>Privacy Policy</p> <p>Cookie Policy</p> <p>Sitemap</p>	<p>Safety Data Sheet</p> <p>Instructions For Use</p> <p>Compliancy & Transparency Kerr</p> <p>Compliancy & Transparency Ervista</p>	<p>GET CONNECTED WITH EMAILS</p> <p>f t i in e</p> <p>©2022 Kerr Corporation</p>
--	---	--