


PLEXUS® P4000/P4001

O₂ZONED™ LAL LATERAL ROTATION

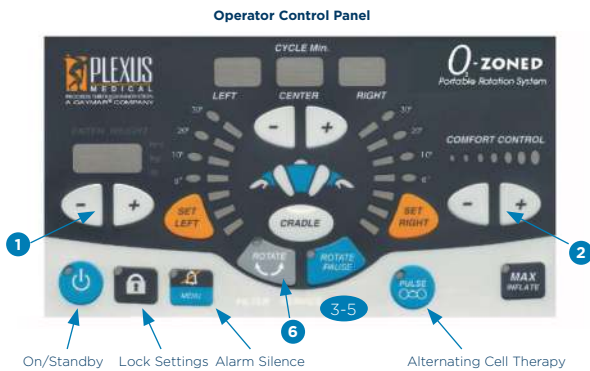

Special Features:

- Press **ROTATE PAUSE** to temporarily pause. Press again to turn off pause.
- Press **CRADLE** to help re-position patient in the center of the mattress.
- Press **MAX INFLATE** to rapidly create a firm mattress surface.
- Press **PULSE** to activate alternating pressure therapy.
- **CPR DEFLATE**: Turn and pull to disconnect hose at the side of control unit for rapid deflation.


- 1 **Enter patient's weight:** Press +/- keys at left of pump to set.
- 2 **Adjust comfort control** based upon patient's comfort, and if additional support is needed. Press +/- key at right of pump to set.
- 3 **Set center position hold/dwell time:** Press +/- keys at center of pump to set.
- 4 **Set left turn angle and hold/dwell time:** Press SET LEFT button until proper turn angle is indicated by light. While numerals are still blinking, press +/- buttons to set dwell time.
- 5 **Set right turn angle and hold/dwell time:** Press SET RIGHT, then +/- buttons as in #3 above.
- 6 **Press Rotate to initiate the turn.**

Please see operator's manual for further information.
For further questions, please call your local Agility office or call **1.800.847.7368**.

©2018 Agility Health, Inc. PLEXUS.P4000/P4001.1018

SPANAMERICA PRESSURE GUARD EASY AIR® LR

LOW AIR-LOSS, ALTERNATING PRESSURE AND ROTATION MATTRESS

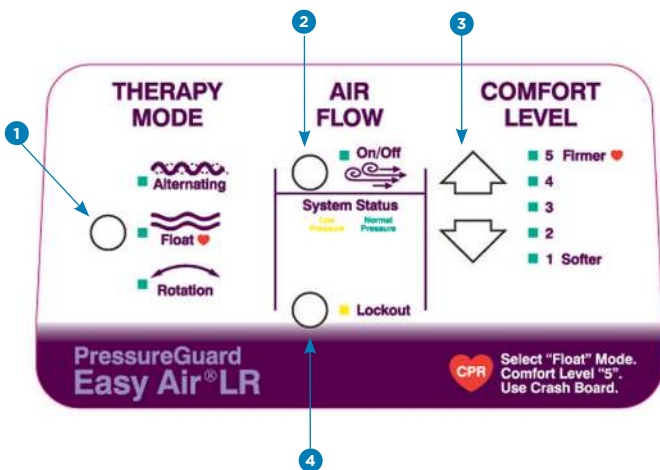

- 1 **Therapy Mode:** Pressing button will toggle between the following therapy mode options:
 - **Alternating:** Provides cyclical tissue loading and unloading.
 - **Float:** Maintains a chosen pressure despite changes to patient position.
 - **Rotation:** Gentle turning in left/center/right pattern.
- 2 **Air Flow:** Provides microclimate management (low air-loss).
- 3 **Comfort Level:** Start in lowest setting and adjust for comfort.
- 4 **Lock Out:** Press and hold for 4 seconds to engage/disengage.

CPR: Select FLOAT mode, comfort level 5 and utilize crash board.

Head of bed elevation above 30: To minimize risk of patient falls, always choose alternating or float mode if head of bed is elevated above 30 degrees.

Please see operator's manual for further information at www.spanamerica.com. For further questions, please call your local Agility office or call **1.800.847.7368**.