



Dominion Aesthetic Technologies, Inc.

eon™ FR OPERATION MANUAL



Copyright © Dominion Aesthetic Technologies, Inc. Printed in USA. All rights reserved.
The contents of this publication may not be reproduced in any form without the explicit permission of
Dominion Aesthetic Technologies, Inc.



Foreword

eon™ FR (1064nm laser) system will be an asset to your practice and will provide you with significant advantages of your aesthetic practice. Our team has over 40 years of experience designing medical laser equipment. We have a deep understanding of the aesthetic physicians' needs and provide a laser-based solution that excels in quality, efficacy and reliability.

eon FR has been developed and is manufactured according to the most stringent quality, safety requirements and standards.

eon FR provides the aesthetic physician with the opportunity to offer their patients precise, gentle and effective treatment. We believe the following advantages will benefit your practices and patients:

- Greater patient comfort and satisfaction.
- Non-contact treatment.
- Quicker, more efficient treatment.
- Higher precision and control

The purpose of this Operation Manual is to provide required information on characteristics of eon FR and its operation. We strongly recommend carefully reading and studying the entire contents of this manual before attempting to operate the device. Please note the various warnings that are provided to ensure a maximum life span of your system and to safeguard the patient, medical personnel and yourself.

In the event you should have any questions or comments regarding the eon FR, we invite you to contact us by email at info@dominionaesthetic.com. Alternatively, we can be contacted through the contact details provided in Section 1.1, Corporate Contact Information.

We know that you will enjoy your eon FR laser system and the added value and advantages it will bring to your practice.

Dominion Aesthetic Technologies, Inc. *

eon™ FR

** Dominion Aesthetic Technologies, Inc. is referred to as Dominion in this document.*



Table of Contents

1	Introduction	7
1.1	Corporate Contact Information	7
1.2	Copyright Notice	7
1.3	Overview of Conventions Used in Manual	8
1.4	Overview of General Precautions	9
1.5	Recommendation of Laser Safety Officer	9
1.6	Software Version	9
2	Clinical Indications for Use	10
2.1	Intended Use	10
2.2	Availability	10
2.3	Contra Indications	10
2.4	Possible Side Effects	10
2.5	Service	11
2.6	Duty Cycle	11
3	Safety	12
3.1	Laser Classification	12
3.1.1	Class IV Lasers and Laser Systems	12
3.2	Laser Safety	13
3.2.1	Safety Program	13
3.2.2	Laser Safety Officer	13
3.2.3	Eye Protection	13
3.2.4	Required Users of Safety Glasses	13
3.2.5	Importance of Wearing Safety Glasses	14
3.2.6	Safety Glasses	14
3.2.7	Using Safety Glasses Correctly	14
3.2.8	Laser Safety Calculations:	16
3.2.9	Beam Divergence:	16
3.2.10	Laser Calibration	16
3.2.11	Laser System Integrity Check	16



3.2.12	Combustible chemicals and gases	17
3.2.13	Laser Safety Warning Signs	18
3.3	Electrical Safety	19
3.3.1	Electromagnetic Interference	20
3.3.2	Electric Safety	21
3.3.3	Compliance	22
3.4	Use Environment	25
3.4.1	Device Placement	25
3.5	Federal Regulation Compliance	25
3.6	Laser Safety Supervision	25
3.7	Warnings	25
3.8	Precautions	27
3.9	Safety Features on Device	28
3.10	Laser Safety Checklist	29
3.11	Sources for Additional Information and Assistance on Laser Safety	29
4	Product Description	30
4.1	General Overview	30
4.2	Labeling	36
4.3	Labels	39
4.4	Symbols	41
5	Instructions for Use	42
5.1	Usage Prerequisites	42
5.2	Laser Lipolysis Treatment	43
5.2.1	Patient Positioning	43
5.2.2	Treatment Parameters	43
5.2.3	Fitzpatrick Scale Skin Types	44
5.3	Software User Interface	44
5.3.1	System Initialization	44
5.3.2	Treatment Ready	46
5.3.3	Treatment Type	47



5.3.4	Treatment Area	47
5.3.5	Treatment Size	47
5.3.6	Treatment Confirmation	47
5.3.7	Laser Safety Check.....	49
5.3.8	Laser Enable	50
5.3.9	Treatment Active	52
5.3.10	Treatment Completion.....	54
5.3.11	Device Shutdown	55
5.4	Software Warning Messages	56
5.4.1	Software Warning Message #1	56
5.4.2	Software Warning Message #2.....	57
5.4.3	Software Warning Message #3.....	58
5.4.4	Software Warning Message #4.....	59
5.4.5	Software Warning Message #5.....	60
5.4.6	Software Warning Message #6.....	61
5.4.7	Software Warning Message #7.....	62
5.4.8	Software Warning Message #8.....	63
5.4.9	Software Warning Message #9.....	64
6	Troubleshooting.....	65
7	Maintenance	66
7.1	Cleaning	66
7.2	Storage.....	67
8	Service	68
9	Warranty Certificate	69
9.1	Standard Warranty Terms.....	69
9.2	Standard Warranty Conditions	69
9.3	Extended Warranties	70



Table of Figures

Figure 4-1 – eon™ FR – Front View	32
Figure 4-2 – System Power Control and Emergency ON/OFF Switch	33
Figure 4-3 – eon™ FR– Rear View	34
Figure 4-4 – Powered Articulated Arm and Treatment Head.....	35
Figure 4-5 – System Power Control and Emergency ON/OFF Switch.....	36
Figure 4-6 – eon™ FR Label Locations.....	37
Figure 4-7 – eon FR Label Locations	38
Figure 5-1: Top-View of the Device in Relation to the Patient.....	43
Figure 5-2: System Initiation Screen	45
Figure 5-3: Treatment Ready Screen	46
Figure 5-4: Laser Safety Check Screen	49
Figure 5-5: Laser Enable Screen.....	50
Figure 5-6: Treatment Active Screen	52
Figure 5-7: Treatment Completion Message	54
Figure 5-8: Device Shutdown Confirmation.....	55
Figure 5-9: Software Warning Message #1.....	56
Figure 5-10: Software Warning Message #2.....	57
Figure 5-11: Software Warning Message #3.....	58
Figure 5-12: Software Warning Message #4.....	59
Figure 5-13: Software Warning Message #5.....	60
Figure 5-14: Software Warning Message #6.....	61
Figure 5-15: Software Warning Message #7.....	62
Figure 5-16: Software Warning Message #8.....	63
Figure 5-17: Software Warning Message #9.....	64