

Congratulations on your purchase of a **FLATULENCE DEODORIZER™- Reusable Pad**.

The material we use is a high- tech filtering media (16) and was originally developed by the British Chemical Defense Establishment, to provide protection to its soldiers against nerve gas and other toxic vapors associated with chemical warfare. Your **FLAT-D** will eliminate any embarrassment brought on by odors associated with flatulence gas.


**Intended Use:** The pad is to be worn inside the underwear and it will eliminate odors associated with flatulence.

**Application:** Your **FLAT-D** may be used daily or on an "as needed" basis. A **FLAT-D** can be washed repeatedly and still maintain its effectiveness. A **FLAT-D** may last several weeks depending on use and laundering. You may use it with or without the adhesive tape. Many keep the **FLAT-D** in a pocket or purse until ready to use, then immediately install it. The pad is placed inside underwear. The large circular part is the backside. The narrow section goes between your legs.

**Custom fit:** You may customize the size of your **FLAT-D** with scissors. The **FLAT-D** is manufactured in a "one size fits all" configuration. You may wish to cut the narrow section or the round area to fit your size. After customizing your **FLAT-D**, you'll need to rinse it in cold water to remove any carbon particles that may have been created.

**Directions: (PRIOR TO WEARING YOUR FLAT-D)**

- ▶ Rinse **FLAT-D** in cold water and let dry (Optional)  
(This will eliminate any carbon dust and will reduce the stiffness of the **FLAT-D**)
- To secure your **FLAT-D** to your underwear- (Use the double-sided tape)
- ▶ Place the **FLAT-D** on a flat horizontal surface. *Both sides are the same.*
- ▶ Remove liner from one side of the double-sided tape. (use tape at own risk)
- ▶ Place sticky tape (28) down in the center of the **FLAT-D** (18)(as in the picture)
- ▶ Take the other liner off the exposed back side of the tape
- ▶ Place **FLAT-D**, tape side down, on the inside of the panties or briefs in the desired location. (centered around the area of the anus) (see picture below)

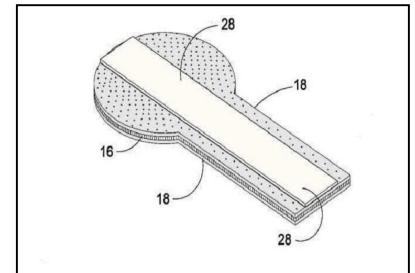

**Wash Instructions:** HAND WASH your **FLAT-D** by filling a sink with cold water and mix in a small amount of a mild detergent, like Woolite or Borax, into the water. Scrub the exterior surface of the **FLAT-D** cloth, then rinse it in clean cold water. Washing the pad only cleans the exterior surface and not the inside black charcoal cloth. **DO NOT place the soap directly on the pad as this will ruin the activated charcoal cloth.** You do not have to remove the tape to wash the pad, in fact we recommend you leave the tape on while washing and place the new tape over the old one. Removing the tape from the **FLAT-D** may remove some of the cloth. For the next use, simply place the new strip of tape over the old one.

**Drying Instructions:** Hang or lay on towel to dry for several hours. You may also use a hair blow dryer, on medium heat, to dry the product. The **Flat-D** must be thoroughly dry to be effective.

**IMPORTANT:** The **FLAT-D** pad must form a seal between the buttocks and the undergarment in order to be effective. The flatulence gas must pass through the **FLAT-D** pad in order to eliminate the odor. The **FLAT-D** may be used with panties, briefs and panty hose. The **FLAT-D** will not be effective if worn with boxer type underwear. The **FLAT-D** may fold over so make sure it is laying flat before using. It should be secured inside the underwear with provided tape.

**Disposal:** Throw away in regular garbage. **DO NOT** flush your **FLAT-D** down the toilet, as product is **not** biodegradable.

**Warning:** For external use only. If irritation develops, discontinue use. Carbon dust may cause eye irritation. If carbon dust gets into the eye, flush thoroughly with water.

**Active Ingredient:** Exterior cloth is a polyester fabric. The middle black cloth is Activated Charcoal, which is an inert, non-toxic material. The provided double-sided tape is a rubber based adhesive.

**Returns:** The product is guaranteed to work as it has been tested extensively. When used properly you will find it very helpful. We will accept returns of unopened product within 10 days. Contact us if you have questions or problems as we want to help you be successful with the use of the product.

**Orders:** Additional products and extra tape are available through our web site  
Email or call us toll free.

**FLAT-D Innovations, Inc.**

P.O. Box 10342  
Cedar Rapids, IA 52410-0342  
Phone: 319-447-4840  
FAX: 610-300-8992  
Toll Free: 1-866-354-0056  
Website: [www.flat-d.com](http://www.flat-d.com)  
Email: [info@flat-d.com](mailto:info@flat-d.com)

