



User Manual

English

Release 1.0

# Collaboration Live

**PHILIPS**



# Contents

- 1 Read This First..... 5**
  - Intended Audience ..... 6
  - Intended Use ..... 6
  - Regulatory Representatives ..... 7
  - Customer Service..... 8
- 2 Safety ..... 9**
  - Warnings and Cautions ..... 9
- 3 Collaboration Live Overview ..... 11**
  - Collaboration Live Accounts ..... 11
  - Privacy ..... 12
- 4 Preparing to Use Collaboration Live ..... 13**
  - Network Requirements ..... 13
  - Setting Up the Ultrasound System ..... 13
    - Finding the System Serial Number and Material Number ..... 14
  - Setting Up the IIT Reacts Client ..... 14
- 5 Using Collaboration Live on the Ultrasound System ..... 15**
  - Collaboration Live Controls ..... 15
  - Collaboration Live Icons ..... 16
  - Starting Collaboration Live on the Ultrasound System ..... 18
  - Starting a Chat Session ..... 18
  - Viewing a New Message or Answering a Call ..... 18
  - Making an Audio Call ..... 19
  - Sharing the Ultrasound Display ..... 19

4535 620 37721\_A/795 \* FEB 2020

Philips

Enabling Remote Takeover.....	20
Sharing Video .....	21
Displaying Patient Information to the Remote Viewer .....	22
Collaboration Live Settings.....	22
Collaboration Live Automatic Backup and Restore .....	22
<b>6 Using Collaboration Live on a Remote Device.....</b>	<b>23</b>
Reacts Client Controls .....	24
Reacts Client Icons.....	25
Starting Collaboration Live on the Remote Device.....	26
Starting a Chat Session .....	27
Viewing a New Message.....	27
Answering a Call .....	28
Making a Call .....	28
Viewing the Shared Display .....	29
Controlling the Ultrasound System Remotely .....	29